



地址/Add : 中国浙江省杭州市临安区玲珑工业园锦溪南路1288号
No.1288 South Jinxi Road, Linglong Industry Park, Lin'an District,
Hangzhou City, Zhejiang Province, 311301, China.
电话/Tel : +86-571-23652388
传真/Fax : +86-571-23651000
网址/Web : <http://www.fushanhz.com>
<http://kingmedical.en.alibaba.com>
邮箱/Email : export@fushanhz.com



杭州富善医疗器械有限公司
Hangzhou Fushan Medical Appliances Co., Ltd.

公司简介 Company Profile

杭州富善医疗器械有限公司创立于2005年7月，坐落在杭州市西郊风景优美的临安区玲珑工业园内。

公司占地面积13000平方米，按照GMP要求设计建造了多个万级和十万级净化车间。车间配备了多套精密的生产和检验设备，可以生产硅橡胶，聚氨酯，聚氯乙烯等医用高分子材料制作的医用耗材。

同时，公司设计建造了专业的实验室，方便进行无菌、理化和电气试验等。公司配备了符合CE标准的环氧乙烷灭菌柜，可以在公司内实现产品的灭菌操作并进行灭菌确认，为客户提供精确的灭菌验证报告。

公司生产销售产品主要包括：呼吸麻醉类、肠胃营养类、创伤引流类、泌尿及妇科类相关耗材。所有产品都已获得欧盟CE认证，部分产品在美国FDA备案。公司还为广大客户提供专业的OEM和ODM服务。

“规范、严谨、专注、创新”一直是公司发展的指导方针。富善人始终致力于关怀人类健康生活，提供优质医疗用品。

Hangzhou Fushan Medical Appliances Co., Ltd. was established in July 2005. It locates at Lin'an Linglong Industrial Park in the west of Hangzhou.


The factory of Fushan Medical covers an area of 13,000 sqms. Both class 10,000 and class 100,000 clean workshops are designed and constructed according to GMP standards. With various precise manufacture and inspection facilities, Fushan provides different kinds of medical devices by silicone, polyurethane (PU) and polyvinyl chloride (PVC).

Meanwhile, Fushan designed and built professional labs for sterile, physicochemical and electrical tests. It also equipped with CE verified ETO sterilization chamber. Sterilization verification report can be issued as request.

Fushan manufactures and sells various kinds of products, which refers to anesthesia series, gastroenterology series, surgical drainage series, urology and gynecology series etc. All products are granted with CE certificate, and most of them are registered in FDA. Fushan also provides OEM/ODM service to the market.

With the philosophy of "standard, preciseness, concentration and innovation", Fushan is always keeping providing high grade medical devices to the world.



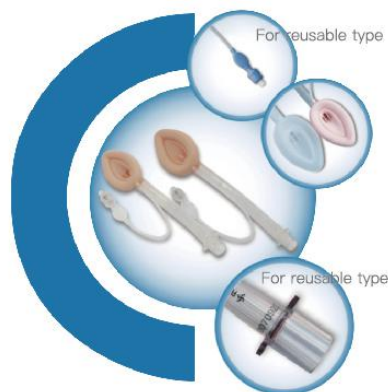

一次性硅胶喉罩

Disposable Silicone Laryngeal Mask Airway

- 100%采用医用硅胶生产，具有良好的生物相容性，无毒无刺激。

Made of 100% medical grade silicone, with good bio-compatibility, non-toxic.

重复性硅胶喉罩

Reusable Silicone Laryngeal Mask Airway

- 经高压蒸汽灭菌可重复使用40次，每个喉罩都配有一个独特的序列号和对应的灭菌记录卡。Suitable for autoclave sterilization. Reusable up to 40 times. With trackable serial number.
- 提供带格栅和无格栅罩囊供客户选择；提供透明罩囊和肉色罩囊供客户选择。
Cuff bar: with/without bar for choice.
Cuff color: transparent/pink for choice.

产品编号 Product No.		规格 Size	罩囊最大充气量 (ML) Maximum Cuff Inflation Volumes(ML)	放气板颜色 Deflating Cap Color	建议适用患者体重 Suggested Patient Weight (KG)
Disposable	Reusable				
LM1004S1	LM1004S2	1	4	粉色/Pink	0-5
LM1507S1	LM1507S2	1.5	7	蓝色/Blue	5-10
LM2010S1	LM2010S2	2	10	绿色/Green	10-20
LM2514S1	LM2514S2	2.5	14	橙色/Orange	20-30
LM3020S1	LM3020S2	3	20	红色/Red	30-50
LM4030S1	LM4030S2	4	30	黄色/Yellow	50-70
LM5040S1	LM5040S2	5	40	紫色/Purple	70-100

一次性加强型硅胶喉罩

Disposable Reinforced Silicone Laryngeal Mask Airway

- 采用进口的医用级硅胶生产，气管经过钢丝加强，可以承受头部和颈部的压力，防止气道扭转，压扁或堵塞。


Made of imported medical grade silicone, with stainless steel spring. The airway is kink-resistant with reinforced tube.

重复性加强型硅胶喉罩

Reusable Reinforced Silicone Laryngeal Mask Airway

- 经高压蒸汽灭菌可重复使用40次，每个喉罩都配有一个独特的序列号和对应的灭菌记录卡。Suitable for autoclave sterilization. Reusable up to 40 times. With trackable serial number.
- 提供带格栅和无格栅罩囊供客户选择；提供透明罩囊和肉色罩囊供客户选择。
Cuff bar: with/without bar for choice.
Cuff color: transparent/pink for choice.

产品编号 Product No.		规格 Size	罩囊最大充气量 (ML) Maximum Cuff Inflation Volumes(ML)	放气板颜色 Deflating Cap Color	建议适用患者体重 Suggested Patient Weight (KG)
Disposable	Reusable				
LM1004S1R	LM1004S2R	1	4	粉色/Pink	0-5
LM1507S1R	LM1507S2R	1.5	7	蓝色/Blue	5-10
LM2010S1R	LM2010S2R	2	10	绿色/Green	10-20
LM2514S1R	LM2514S2R	2.5	14	橙色/Orange	20-30
LM3020S1R	LM3020S2R	3	20	红色/Red	30-50
LM4030S1R	LM4030S2R	4	30	黄色/Yellow	50-70
LM5040S1R	LM5040S2R	5	40	紫色/Purple	70-100

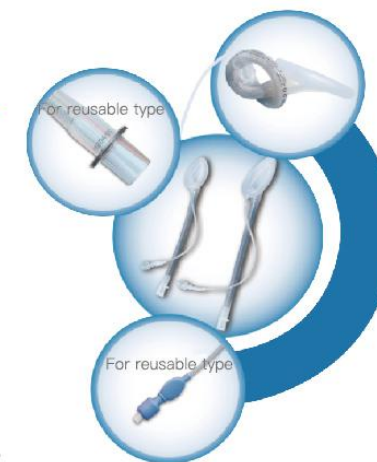

一次性PVC喉罩 Disposable PVC Laryngeal Mask Airway

- ◆ 采用医用级PVC生产。
Made of medical grade PVC.
- ◆ 罩囊外形符合人体咽喉部解剖结构，最大限度减少对患者咽喉部的刺激，气密性好。
The cuff is designed according to anatomical structure of human pharynx and larynx, which reduces stimulation to patient and leakage risk.



产品编号 Product No.	规格 Size	罩囊最大充气量 (ML) Maximum Cuff Inflation Volumes(ML)	放气板颜色 Deflating Cap Color	建议适用患者体重 Suggested Patient Weight (KG)
LM1004P	1	4	粉色/Pink	0-5
LM1507P	1.5	7	蓝色/Blue	5-10
LM2010P	2	10	绿色/Green	10-20
LM2514P	2.5	14	橙色/Orange	20-30
LM3020P	3	20	红色/Red	30-50
LM4030P	4	30	黄色/Yellow	50-70
LM5040P	5	40	紫色/Purple	70-100

一次性PVC硅胶喉罩 Disposable PVC-Silicone Combined Laryngeal Mask Airway

- ◆ 喉罩气管采用医用级PVC生产，罩囊采用医用级硅胶生产。
Its tube is made of medical grade PVC, and its cuff is made of medical grade silicone.
- ◆ 结合硅胶喉罩和PVC喉罩的特点，兼顾舒适性及经济性。
It combines the advantage of PVC LMA and silicone LMA, with good comfortability and economy.




产品编号 Product No.	规格 Size	罩囊最大充气量 (ML) Maximum Cuff Inflation Volumes(ML)	放气板颜色 Deflating Cap Color	建议适用患者体重 Suggested Patient Weight (KG)
LM1004P1	1	4	粉色/Pink	0-5
LM1507P1	1.5	7	蓝色/Blue	5-10
LM2010P1	2	10	绿色/Green	10-20
LM2514P1	2.5	14	橙色/Orange	20-30
LM3020P1	3	20	红色/Red	30-50
LM4030P1	4	30	黄色/Yellow	50-70
LM5040P1	5	40	紫色/Purple	70-100

一次性加强型硅胶气管插管 Disposable Reinforced Silicone Endotracheal Tube


- ◆ 100%采用医用级硅胶生产；管体柔软透明。
Made of 100% medical grade silicone, the tube is soft and clear.
- ◆ 带高容量低压力气囊。
With high volume and low pressure silicone cuff.
- ◆ 通气管经过弹簧加强，可防止管体压扁、扭曲。
The main tube is reinforced with stainless steel spring to prevent crushing and kinking.
- ◆ 带墨菲孔。
With Murphy eye design.

产品编号 Product No.	规格 Size	产品编号 Product No.	规格 Size	产品编号 Product No.	规格 Size
ET40SR	4.0	ET65SR	6.5	ET90SR	9.0
ET45SR	4.5	ET70SR	7.0	ET95SR	9.5
ET50SR	5.0	ET75SR	7.5	ET100SR	10.0
ET55SR	5.5	ET80SR	8.0		
ET60SR	6.0	ET85SR	8.5		

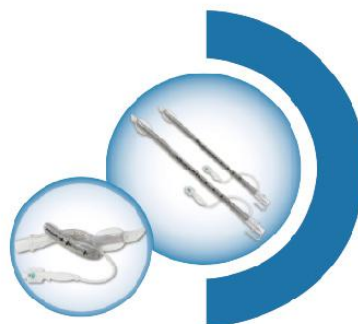

PVC气管插管 PVC Endotracheal Tube

- ◆ 采用医用级PVC生产，管体透明度高。
Made of medical grade PVC, the tube is transparent and clear.
- ◆ X光造影线贯穿管身。
X-ray opaque line through the whole tube.
- ◆ 高容量低压力气囊可供选择。
High volume and low pressure cuff is available for choice.
- ◆ 带墨菲孔。
With Murphy eye design.

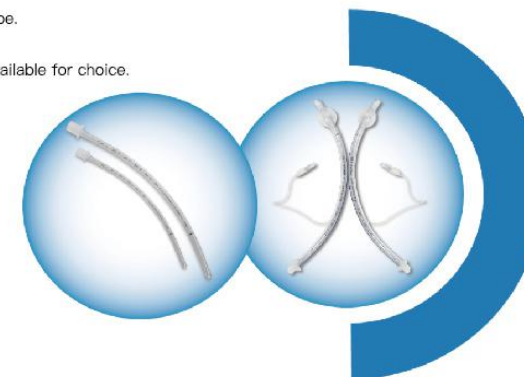

重复性加强型硅胶气管插管 Reusable Reinforced Silicone Endotracheal Tube

- ◆ 100%采用医用级硅胶生产；管体柔软透明。
Made of 100% medical grade silicone, the tube is soft and clear.
- ◆ 带高容量低压力气囊。
With high volume and low pressure silicone cuff.
- ◆ 通气管经过弹簧加强，可防止管体压扁、扭曲。
The main tube is reinforced with stainless steel spring to prevent crushing and kinking.
- ◆ 带墨菲孔。
With Murphy eye design.
- ◆ 重复使用达10次。
Reusable up to 10 times.

产品编号 Product No.	规格 Size	产品编号 Product No.	规格 Size	产品编号 Product No.	规格 Size
ET40S2R	4.0	ET65S2R	6.5	ET90S2R	9.0
ET45S2R	4.5	ET70S2R	7.0	ET95S2R	9.5
ET50S2R	5.0	ET75S2R	7.5	ET100S2R	10.0
ET55S2R	5.5	ET80S2R	8.0		
ET60S2R	6.0	ET85S2R	8.5		



产品编号 Product No.		规格 Size	产品编号 Product No.		规格 Size
Without Cuff	With Cuff		Without Cuff	With Cuff	
ET20P	/	2.0	ET65P	ET65PC	6.5
ET25P	ET25PC	2.5	ET70P	ET70PC	7.0
ET30P	ET30PC	3.0	ET75P	ET75PC	7.5
ET35P	ET35PC	3.5	ET80P	ET80PC	8.0
ET40P	ET40PC	4.0	ET85P	ET85PC	8.5
ET45P	ET45PC	4.5	ET90P	ET90PC	9.0
ET50P	ET50PC	5.0	ET95P	ET95PC	9.5
ET55P	ET55PC	5.5	ET100P	ET100PC	10.0
ET60P	ET60PC	6.0			

口腔预成型PVC气管插管 Oral Preformed PVC Endotracheal Tube


- 采用医用级PVC生产，管体透明度高。
Made of medical grade PVC, the tube is transparent and clear.
- X光造影线贯穿管身。
X-ray opaque line through the whole tube.
- 大容量低气压气囊可供选择。
High volume and low pressure cuff is available for choice.
- 带墨菲孔。
With Murphy eye design.
- 口腔预成型，经口使用最佳。
With oral preformed design, it is better to use for oral insertion.



产品编号 Product No.		规格 Size	产品编号 Product No.		规格 Size
Without Cuff	With Cuff		Without Cuff	With Cuff	
OT20P	/	2.0	OT65P	OT65PC	6.5
OT25P	OT25PC	2.5	OT70P	OT70PC	7.0
OT30P	OT30PC	3.0	OT75P	OT75PC	7.5
OT35P	OT35PC	3.5	OT80P	OT80PC	8.0
OT40P	OT40PC	4.0	OT85P	OT85PC	8.5
OT45P	OT45PC	4.5	OT90P	OT90PC	9.0
OT50P	OT50PC	5.0	OT95P	OT95PC	9.5
OT55P	OT55PC	5.5	OT100P	OT100PC	10.0
OT60P	OT60PC	6.0			

鼻腔预成型PVC气管插管 Nasal Preformed PVC Endotracheal Tube

- 采用医用级PVC生产，管体透明度高。
Made of medical grade PVC, the tube is transparent and clear.
- X光造影线贯穿管身。
X-ray opaque line through the whole tube.
- 大容量低气压气囊可供选择。
High volume and low pressure cuff is available for choice.
- 带墨菲孔。
With Murphy eye design.
- 鼻腔预成型，经鼻使用最佳。
With nasal preformed design, it is better to use for nasal insertion.

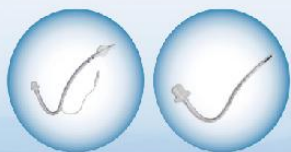

产品编号 Product No.		规格 Size	产品编号 Product No.		规格 Size
Without Cuff	With Cuff		Without Cuff	With Cuff	
NT20P	/	2.0	NT65P	NT65PC	6.5
NT25P	NT25PC	2.5	NT70P	NT70PC	7.0
NT30P	NT30PC	3.0	NT75P	NT75PC	7.5
NT35P	NT35PC	3.5	NT80P	NT80PC	8.0
NT40P	NT40PC	4.0	NT85P	NT85PC	8.5
NT45P	NT45PC	4.5	NT90P	NT90PC	9.0
NT50P	NT50PC	5.0	NT95P	NT95PC	9.5
NT55P	NT55PC	5.5	NT100P	NT100PC	10.0
NT60P	NT60PC	6.0			

加强型PVC气管插管 Reinforced PVC Endotracheal Tube

- 采用医用级PVC生产，管体透明度高。
Made of medical grade PVC, the tube is transparent and clear.
- 通气管经过弹簧加强，可防止管体压扁、扭曲。
The main tube is reinforced with stainless steel spring to prevent crushing and kinking.
- 大容量低气压气囊可供选择。
High volume and low pressure cuff is available for choice.
- 带墨菲孔。
With Murphy eye design.



产品编号 Item No.		规格 Size	产品编号 Product No.		规格 Size
Without Cuff	With Cuff		Without Cuff	With Cuff	
ET20PR	/	2.0	ET65PR	ET65PCR	6.5
ET25PR	ET25PCR	2.5	ET70PR	ET70PCR	7.0
ET30PR	ET30PCR	3.0	ET75PR	ET75PCR	7.5
ET35PR	ET35PCR	3.5	ET80PR	ET80PCR	8.0
ET40PR	ET40PCR	4.0	ET85PR	ET85PCR	8.5
ET45PR	ET45PCR	4.5	ET90PR	ET90PCR	9.0
ET50PR	ET50PCR	5.0	ET95PR	ET95PCR	9.5
ET55PR	ET55PCR	5.5	ET100PR	ET100PCR	10.0
ET60PR	ET60PCR	6.0			

麻醉呼吸类

ANESTHESIA SERIES

麻醉呼吸类

ANESTHESIA SERIES

氧气面罩 Oxygen Mask

- ◆ 采用医用级PVC生产，面罩柔软且透明度高。
Made of medical grade PVC, the mask is soft and clear.
- ◆ 配备可调节鼻夹。供氧管长度可定制。
Nose clip is adjustable. The length of oxygen tube can be customized.


产品编号 Product No.	型号 Type
102MM-E	XL: Adult Elongated
102MM	L: Adult Standard
222MM-E	M: Pediatric Elongated
222MM	S: Pediatric Standard



吸痰管 Suction Tube

- ◆ 采用医用级PVC生产，管体透明度高。
Made of medical grade PVC, the tube is transparent and clear.
- ◆ 多种类型接头可供选择。
Different kinds of connectors are available.


产品编号 Product No.	规格 (Fr) Size(Fr)	色标号 Color Code	长度 (mm) Length(mm)	备注 Remark
SC05	5	灰色/Gray	450	Cap-cone type Plain type
SC06	6	浅绿色/Light Green		
SC08	8	浅蓝色/Light Blue		
SC10	10	黑色/Black		
SC12	12	白色/White		
SC14	14	绿色/Green		
SC16	16	橙色/Orange		
SC18	18	红色/Red		
SC20	20	黄色/Yellow		

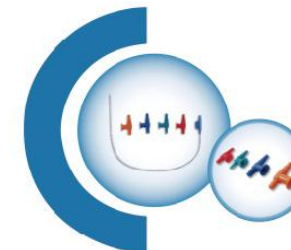

雾化面罩 Nebulizer Mask

- ◆ 采用医用级PVC生产，面罩柔软且透明度高。
Made of medical grade PVC, the mask is soft and clear.
- ◆ 配备6ml雾化罐。
6ml nebulizer jar is available.

产品编号 Product No.	型号 Type
AK06AE	XL: Adult Elongated
AK06AS	L: Adult Standard
AK06PE	M: Pediatric Elongated
AK06PS	S: Pediatric Standard

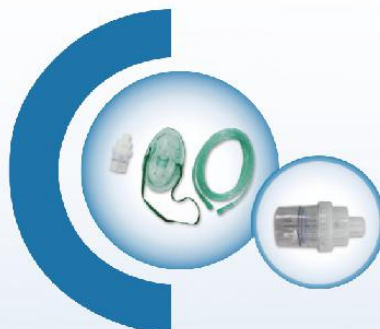

吸引连接管 Suction Connecting Tube

- ◆ 采用医用级PVC生产，管体透明度高。
Made of medical grade PVC, the tube is transparent and clear.
- ◆ 管体长度可定制。常规长度有：1.8米，2米，2.5米，3米等。
The length of tube can be customized. 1.8m, 2.0m, 2.5m and 3.0m are regular lengths for choice.

产品编号 Product No.	尺寸 (m) Size(m)	产品编号 Product No.	尺寸 (m) Size(m)	备注 Remark
ST18T	3/16" 1.8m	ST18H	1/4" 1.8m	Customized Length
ST20T	3/16" 2.0m	ST20H	1/4" 2.0m	
ST25T	3/16" 2.5m	ST25H	1/4" 2.5m	
ST30T	3/16" 3.0m	ST30H	1/4" 3.0m	



SLIPA-3G 硅胶双腔免充气喉罩



型号 Size	患者体重 Patient Weight	置入引流管最大型号 Maximum Size of Gastric Tube
1.0	0-10 KGS	6Fr
2.0	10-30 KGS	10Fr
3.0	30-50 KGS	12Fr
4.0	50-70 KGS	14Fr
5.0	70-100 KGS	14Fr

【产品特点】Product Features

1. 采用独特的自密封技术，在正压通气时，充盈的气体可使罩囊与咽喉组织更贴合，从而达到良好的密封性。With the unique self-sealing technology, under positive pressure ventilation, the air will make the cuff to fit the patient's pharyngeal cavity better, so as to achieve better sealing performance.
2. 密封压高，压力损伤风险却更低。With higher sealing pressure, but lower risk of pressure damage to patient.
3. 免充气罩囊，结构简单，密封性好，可安全地应用于较长时手术通气。With non-inflation cuff design, its structure is simple and its sealing performance is good. SLIPA-3G can be used for long-term operation ventilation.
4. 专利设计，独特的防反流误吸保护机制：Patented design is adopted, it is with four unique protection mechanism against reflux and aspiration:
 - a. 封闭食道以阻止反流液体。Seal the patient's esophagus to prevent reflux.
 - b. 设有胃液引流通道，可置入胃管。Equipped with gastric drainage channel, gastric tube can be inserted so as to reduce the risk of reflux and aspiration.
 - c. 罩囊内有适当容积的反流液收集腔，可以存储反流到密封囊内的反流液。There is an appropriate volume of reflux collection chamber in the cuff, which can store the reflux liquid.
 - d. 罩囊背面设计有反流物通道，可使大容量的反流液快速排出口腔。There is a reflux channel in the back of cuff, which can discharge large volume of reflux liquid from patient's mouth quickly.

ALMA 硅胶双腔喉罩



【产品特点】Product Features

1. 进口医用级硅胶原料生产，生物相容性好。Made of imported medical grade silicone, ALMA is with very good bio-compatibility.
2. 大容量充气囊，最大程度贴合口咽部结构，减少对咽喉部黏膜的压力，提高患者的舒适度。With high volume cuff design, the cuff of ALMA fits to patient's oropharyngeal structure properly, which reduces the pressure to patient's pharyngeal mucosa, and increases its comfortability.
3. 采用根据人体咽喉部生理弯曲预成型的C型设计，富有韧性，置管更加快捷、准确。The C-shape design based on the physiological bending of human throat makes the insertion easier, faster and more accurate.
4. 设有胃液引流通道，可置入胃管，减少反流误吸的风险。ALMA is equipped with gastric drainage channel. Gastric tube can be inserted through it so as to reduce the risk of reflux and aspiration.
5. 罩囊一体成型，充气囊气道密封压可达35cm水柱。The cuff is integrated and its sealing pressure is over 35cm H₂O.

型号 Size	患者体重 Patient Weight	置入引流管最大型号 Maximum Size of Gastric Tube	罩囊最大充气量 Maximum Volume of Cuff	最大囊内压力 Maximum Pressure inside Cuff
1.0	0-5KG	6Fr	5ml	≤60cmH ₂ O
1.5	5-10KG	8Fr	9ml	≤60cmH ₂ O
2.0	10-20KG	10Fr	12ml	≤60cmH ₂ O
2.5	20-30KG	12Fr	20ml	≤60cmH ₂ O
3.0	30-50KG	14Fr	30ml	≤60cmH ₂ O
4.0	50-70KG	14Fr	40ml	≤60cmH ₂ O
5.0	70-100KG	14Fr	50ml	≤60cmH ₂ O

SLIPA

免充气喉罩

[产品特点] Product Features

1. 专利设计，独特的反流液收集腔，最多可收集50ml反流液。Patented design is adopted, it is equipped with unique reflux collection chamber which is able to collect reflux liquid up to 50ml maximum.
2. 免充气气囊，结构简单，没有气囊压力增大的风险。With non-inflation cuff design, its structure is simple, no risk of cuff pressure increasing.
3. 专利的外形设计，其形状与人体咽喉部的解剖结构完全一致，在尺寸选择正确的情况下可以达到25cm水柱以上的密封压。With patented design, the shape of SLIPA is completely consistent with the anatomical structure of human throat. The sealing pressure reaches more than 25cm H₂O if the size is selected correctly.
4. SLIPA置入到位后有落空感，前端抵在食道开口处，后端卡在鼻咽和软腭之间。定位准确，不易移位。When SLIPA is inserted to a right position, it feels like entering a buckle. Its front end is against the esophageal opening, and the back end is stuck between the nasopharynx and the soft palate. SLIPA can be fixed to a correct position accurately and stably.



SLIPA-2G

双腔免充气喉罩



[产品特点] Product Features

1. SLIPA喉罩的基础上增加胃液引流通道，可置入胃管。On the basis of SLIPA design, a gastric drainage channel is added so as to insert the gastric tube through the airway.
2. 采用更加柔软的医用高分子材料生产，提高患者的舒适度。Made of softer and more flexible medical polymer materials, it makes the patients more comfortable.

Item	Size	35	39	43	47	49	51	53
甲状软骨宽度 Width of Thyroid Cartilage		35mm	39mm	43mm	47mm	49mm	51mm	53mm
对应经典喉罩 Equals to Classic LMA		1.5#	2.0#	2.5#	3.0#	3.5#	4.0#	5.0#
对应公斤体重 Patient Weight		5-10kg	10-20kg	20-30kg	30-40kg	40-60kg	60-70kg	>70kg
置入引流管最大型号* Maximum Size of Gastric Tube		10Fr	10Fr	10Fr	14Fr	14Fr	14Fr	14Fr

*Note: Only for SLIPA-2G



麻醉医生手持SLIPA喉罩臂部，将喉罩前端置入患者口腔，同时将患者头部放置在吸气位置（头部后仰，曲颈）。The anesthesiologist shall hold the body of SLIPA, put the front end of it into patient's mouth, and place patient's head in the aspiratory position (head tilted back, neck bent).



继续向下推进喉罩，同时按压喉罩背部凸起，使其顺利通过门齿。Continue to push SLIPA downward, and press the back bulge of SLIPA so as to make it pass through patient's front teeth smoothly.




托住患者下颌，拇指抵住喉罩跟部。拇指用力将喉罩向下向前推进（过程中有落空感），使其滑入咽喉深处直至跟部卡在鼻咽和软腭之间。Hold patient's jaw and put the thumb against the heel of SLIPA. Push SLIPA downward and forward with force by thumb (there is a feeling like entering a buckle), and make SLIPA slide into the deep throat until the heel is stuck between patient's nasopharynx and soft palate.



麻醉医生可稍用力向前推或略向上提拉喉罩，存在一定阻力时，即表明SLIPA喉罩已处于正确位置。The anesthesiologist can push forward or pull up SLIPA with a little force. When it feels a little resistance, it means that the SLIPA is in the right position.




将SLIPA喉罩与呼吸机连接，检查气密性并保持气道通畅。Connect SLIPA to the ventilator, check the sealing performance and keep the airway unobstructed.

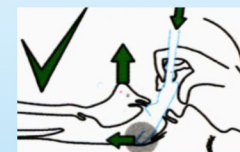

口咽通气道 Oropharyngeal Airway / Guedel Airway

- 采用无毒聚乙烯材料生产。
Made of non-toxic PE.
- 配备不同颜色的色标以区分型号。
Equipped with different color code to identify different sizes.

产品编号 Product No.	规格 Size	色标号 Color Code	长度 (mm) Length(mm)
OA40A	000	粉红/Pink	40
OA50A	00	蓝色/Blue	50
OA60A	0	黑色/Black	60
OA70A	1	白色/White	70
OA80A	2	绿色/Green	80
OA90A	3	黄色/Yellow	90
OA100A	4	红色/Red	100
OA110A	5	浅蓝色/Light Blue	110
OA120A	6	橙色/Orange	120

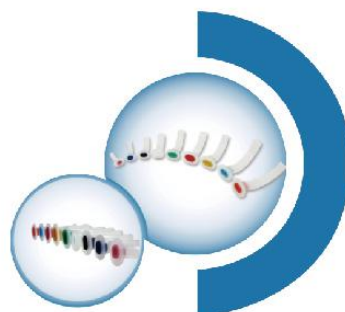

鼻氧管 Oxygen Cannula

- 采用医用级PVC生产，管体柔软且透明度高。
Made of medical grade PVC, the tube is soft and clear.
- 有透明管体和绿色管体可供选择。
Transparent tube and green tube are both available.

产品编号 Product No.	型号 Type
NOC-L	成人/Adult
NOC-M	儿童/Pediatric
NOC-S	婴儿/Infant
NOC-XS	新生儿/Neonate



无针接头 Needle Free Connector

- 采用进口PC材料和医用级硅胶生产，抗感染能力强。
Made of imported PC and medical grade silicone, with high anti-infectivity.
- 密封性好，普通款耐受150KPa压力不漏液，正压款耐受300KPa压力不漏液。
The connector is with high sealability. Standard type connector will not leak under pressure less than 150KPa; positive pressure type connector will not leak under pressure less than 300KPa.
- 6%鲁尔锥度接头设计，方便与市面上的留置针，导管，注射器，输液器等配套使用。
With 6% Luer conical design, it can be used with regular IV cannula, CVC, syringe, infusion set and other kits conveniently.
- 给药和采样无针化，全面保护医务人员，避免职业暴露。
With needle free design for both drug delivery and sampling, it prevent the medical staffs properly from infection damage.



规格 Product No.	5302	5303
分类 Type	普通型 Standard Type	正压封管型 Positive Pressure Type
多路设计 Multi-lumens Design	单路、双路、三路 Single lumen Double lumens Triple lumens	单路、双路、三路 Single lumen Double lumens Triple lumens
延长管长度 (cm) Length of Extension Tube	10-30	10-30
尺寸 Measure	12*24mm	10*36mm
流速 Flow Velocity	220-260ml/min	160-220ml/min

血管通路类

VASCULAR ACCESS SERIES

血管通路类

VASCULAR ACCESS SERIES

一次性使用中心静脉导管包 Disposable Central Venous Catheter Kit

采用符合人体留置医用级专用PU材质，具备优良的生物相容性和化学稳定性优越的弹性，人体温度下自动软化更利于保护体血管组织，免于受损。

Made of medical grade PU material which is specifically used for human body indwelling. It is with excellent biocompatibility and chemical stability, as well as superior elasticity. The material will soften itself automatically to protect the vascular tissue under body temperature.



三角翼形座，该设计减少在固定时与病人皮肤的摩擦，使病人感觉更舒适，专利连接技术更安全更可靠。
The design of Delta wing shape will reduce the friction when it is fixed onto patient's body. It makes patient feel much more comfortable. It is safer and more reliable.

多腔开口流向设计，确保临床同时开展多种药物和药液输注，有效避免药物间的配伍禁忌。全段X光显影，确保安全留置。

The design of multi-lumen ensures that the clinician could inject several kinds of pharmaceuticals at the same time. It will avoid drug incompatibility efficaciously.

The whole tubing could be observed under X-ray, which will ensure the patients' safety when indwelling.

导管末端经过熔融连接方式，连接一段柔软的尖端头型，避免插管过程中以及留置导管过程中对血管造成的损伤。

The distal tip of catheter connects a special soft tip by fusion process. It will avoid the vascular damage when inserting or indwelling the catheter.



导丝
Guide wire

蓝孔注射器
Introducer syringe

导管夹
Catheter clamp

血管扩张器
Vessel dilator

管腔 Lumen Design	导管规格/长度 (cm) Catheter Specification/ Length	扩张器外径 (mm) Dilator OD (mm)	导丝外径/长度 Guide wire (OD)/Length	Y针配置 Y-Shaped Syringe Needle	蓝孔针配置 Blueport/Stop-Wire Introduction Syringe	备注 Remarks
单腔/Single	14Gx16	2.70	0.032x60cm	available		
	14Gx20	2.70	0.032x60cm	available		
	14Gx30	2.70	0.032x70cm	available		
	16Gx16	2.00	0.032x60cm	available		
	16Gx20	2.00	0.032x60cm	available		
	16Gx30	2.00	0.032x70cm	available		
	18Gx13	1.70	0.025x50cm	available		少儿 Pediatric
	18Gx16	1.70	0.025x50cm	available		少儿 Pediatric
18Gx20	1.70	0.025x50cm	available		少儿 Pediatric	
双腔/Double	4F x 8	1.70	0.018x50cm			婴儿 Infant
	4F x 13	1.70	0.018x50cm			婴儿 Infant
	5F x 13	2.00	0.021x50cm			少儿 Pediatric
	5F x 16	2.00	0.021x50cm			少儿 Pediatric
	7F x 16	2.70	0.035x60cm	available		
	7F x 20	2.70	0.035x60cm	available		
	7F x 30	2.70	0.035x70cm	available		
三腔/Triple	4.5F x 10	1.70	0.021x50cm			少儿 Pediatric
	4.5F x 13	1.70	0.021x50cm			少儿 Pediatric
	5F x 10	2.00	0.021x50cm			少儿 Pediatric
	5F x 13	2.00	0.021x50cm			少儿 Pediatric
	5.5F x 13	2.00	0.021x50cm			
	5.5F x 16	2.00	0.021x50cm			
	7F x 16	2.70	0.035x60cm	available		
	7F x 20	2.70	0.035x80cm	available		
	7F x 30	2.70	0.035x70cm	available		



经皮内窥镜胃造瘘管/套装

Percutaneous Endoscopic Gastrostomy (PEG) Tube / Kit

全新的设计


Brand new design

- 根据市场反馈，富善从配件，托盘，到吸塑盒都进行了重新设计。新套装使用更加便捷。

According to the feedback from market, Fushan redesigned the PEG kit totally from components to package. The new kit is more convenient to handle.

- 富善提供两种不同配置的PEG套装：标准装和豪华装。

Both economic pack and luxury pack PEG kit are available for choice.

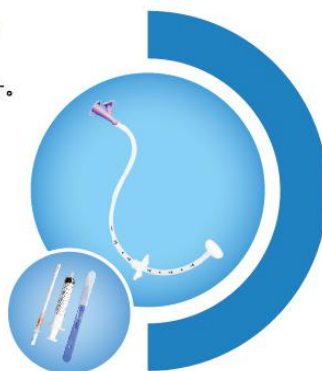

富善的承诺

Fushan's commitment

- 富善医疗致力于开发优质的产品，提供可靠的服务，满足市场的需求。

Fushan provides products with excellent quality and reliable service to meet the requirement of market.

优异的品质

Excellent Quality

- 富善始终坚持着执行最高标准的生产工艺。产品特点包括：

Fushan insists in manufacturing products with the highest standards. The Features of PEG kit include:

- | | |
|---|-------------|
| · Medical Grade Silicone Construction | · 医用级硅胶制造 |
| · External Retention Ring | · 固定圈 |
| · Universal Feeding Port Adapters | · 通用喂食端连接头 |
| · Collapsible Internal Retention Bumper | · 可折叠内部缓冲垫 |
| · Radiopaque Stripe and Bumper | · X光造影线和缓冲垫 |
| · Tubing Clamp | · 导管夹 |
| · Pull Placement Methods | · 牵引替换方法 |

产品编号 Product No.		规格 Size	外径 O.D.(mm)	长度 Length(mm)	备注 Remark
Economic kit	Luxury kit				
PEG-16SE	PEG-16SL	16FR	5.3	600	Length could be customized
PEG-18SE	PEG-18SL	18FR	6.0		
PEG-20SE	PEG-20SL	20FR	6.7		
PEG-22SE	PEG-22SL	22FR	7.3		
PEG-24SE	PEG-24SL	24FR	8.0		

PEG套装（豪华装）

PEG Kit (Luxury Pack)

Traction Removable PULL PEG Tube × 1
 5 Fr Retrieval Snare (1.8mm x 160cm) × 1
 Feeding Adapter × 1
 Feeding Adapter with Luer lock × 1
 Retention Ring × 1
 25Ga Needle × 1
 19/21Ga Needle × 1
 Stainless Steel Scissors × 1
 Stainless Steel Hemostat × 1
 Tubing Clamp × 1
 Water Soluble Lubricant × 1
 10*10 Gauze Sponges × 4
 5*5 Slit Gauze Sponges × 2
 Placement Wire (0.36/0.6mm × 260cm) × 1
 10ml Syringe × 1
 Disposable Scalpel with #11 Blade × 1
 Introducer Needle × 1
 Fenestrated Drape × 1
 Directions for Use × 1


PEG导管
 圈套器
 Y接头
 带鲁尔Y接头
 固定圈
 25G针头
 19/21G针头
 不锈钢手术剪
 不锈钢止血钳
 导管夹
 外科润滑剂
 纱布
 带孔纱布
 导丝
 注射器
 长柄手术刀
 套管针
 洞巾
 使用说明书

PEG套装（标准装）


PEG Kit (Economic Pack)

Traction Removable PULL PEG Tube × 1
 Feeding Adapter × 1
 Feeding Adapter with Luer lock × 1
 Retention Ring × 1
 19/21Ga Needle × 1
 25Ga Needle × 1
 Placement Wire (0.36/0.6mm × 260cm) × 1
 10ml Syringe × 1
 Disposable Scalpel with #11 Blade × 1
 Introducer Needle × 1
 Tubing Clamp × 1
 10*10 Gauze Sponges × 4
 Fenestrated Drape × 1
 Directions for Use × 1

PEG导管
 Y接头
 带鲁尔Y接头
 固定圈
 19/21G针头
 25G针头
 导丝
 注射器
 手术刀
 套管针
 导管夹
 纱布
 洞巾
 使用说明书

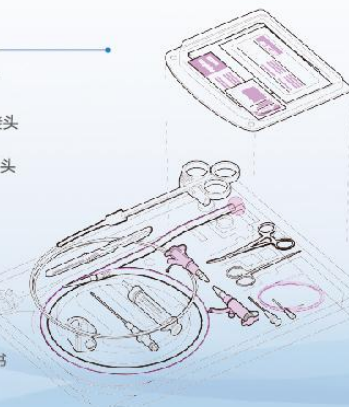

硅胶鼻饲管 Silicone Nasogastric Tube / Feeding Tube

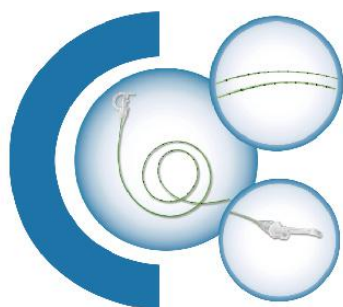

- ◆ 100%采用医用级硅胶生产，管体柔软透明。
Made of 100% medical grade silicone, the tube is soft and clear.
- ◆ X光造影线贯穿管身。
X-ray opaque line through the whole tube.
- ◆ 管体刻度清晰。
Clear depth marks on the tube.
- ◆ 尺寸：5Fr、6.5Fr、8Fr。
Size: 5Fr, 6.5Fr, 8Fr.

硅胶胃造瘘管 Silicone Gastrostomy Tube


- ◆ 100%采用医用级硅胶生产，管体柔软透明。
Made of 100% medical grade silicone, the tube is soft and clear.
- ◆ X光造影线贯穿管身。
X-ray opaque line through the whole tube.
- ◆ 内外粘接球囊，柔软且坚韧。
The balloon is glued to the main tube both inside and outside, it is elastic and flexible.
- ◆ 适用于胃造瘘术的患者。
It is used for gastrostomy operation.

产品编号 Product No.	规格 (Fr) Size(Fr)	色标 Color Code	长度 (mm) Length(mm)	备注 Remark
GT1205	12Fr/5ml	白色/White	240	一次性使用 Disposable
GT1405	14Fr/5ml	绿色/Green		
GT1620	16Fr/20ml	橙色/Orange		
GT1820	18Fr/20ml	红色/Red		
GT2020	20Fr/20ml	黄色/Yellow		
GT2220	22Fr/20ml	紫色/Purple		
GT2420	24Fr/20ml	深蓝/Dark Blue		



硅胶胃管 Silicone Stomach Tube

- ◆ 100%采用医用级硅胶生产，管体柔软透明。
Made of 100% medical grade silicone, the tube is soft and clear.
- ◆ X光造影线贯穿管身。
X-ray opaque line through the whole tube.
- ◆ 管体刻度清晰。
Clear depth marks on the tube.

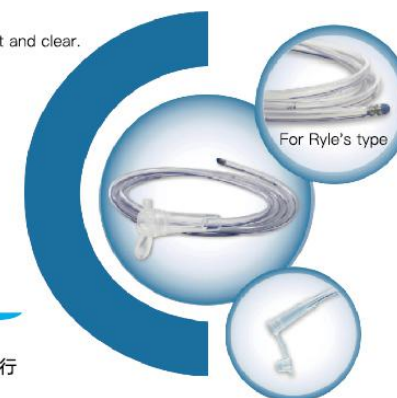


硅胶重头胃管 Silicone Ryle's Stomach Tube

- ◆ 导管底端带不锈钢珠设计，便于导管在指定位置进行有效的留置。
Stainless steel balls are sealed in the tip end; it is easier to indwell the tube at a certain position.

产品编号 Product No.		规格 (Fr) Size(Fr)	长度 (mm) Length(mm)		备注 Remark
Standard	Ryle's		Standard	Ryle's	
ST08S	ST08SR	8	700	700	一次性使用 Disposable
ST10S	ST10SR	10			
ST12S	ST12SR	12	1200	1200	
ST14S	ST14SR	14			
ST16S	ST16SR	16			
ST18S	ST18SR	18			

PU胃管/鼻饲管 PU Stomach Tube / Nasogastric Tube

- ◆ 采用医用级PU生产，管体柔软透明，耐腐蚀性好。
Made of medical grade PU, the tube is soft and clear, with high corrosion resistance.
- ◆ X光造影线贯穿管身。
X-ray opaque line through the whole tube.
- ◆ 管体刻度清晰。
Clear depth marks on the tube.
- ◆ 配备不同颜色的接头以区分不同型号的导管。
Nurse can easily identify different size catheter with different color connectors.

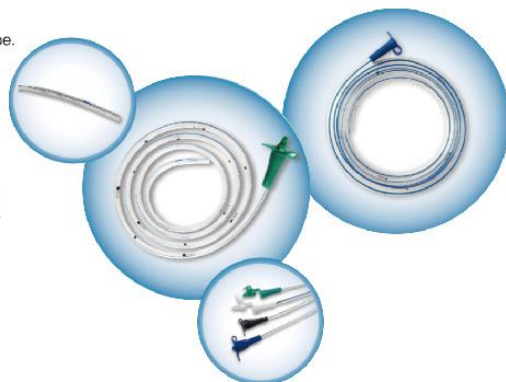

胃管产品编号 Stomach Tube Product No.	规格 (Fr) Size(Fr)	长度 (mm) Length(mm)	鼻饲管产品编号 Nasogastric Tube Product No.	规格 (Fr) Size(Fr)	长度 (mm) Length(mm)
ST08T	8	900	NA40TS	4.0	600
ST10T	10	900	NA50TS	5.0	600
ST12T	12	1250	NA50TL	5.0	1400
ST14T	14	1250	NA65TS	6.5	600
ST16T	16	1250	NA65TL	6.5	1400
ST18T	18	1250	NA80TS	8.0	600
ST20T	20	1250	NA80TL	8.0	1400

PVC胃管 PVC Stomach Tube

- ◆ 采用医用级PVC生产，管体透明度高。
Made of medical grade PVC, the tube is transparent and clear.
- ◆ X光造影线贯穿管身。
X-ray opaque line through the whole tube.
- ◆ 配备不同颜色的接头以区分不同型号的导管。
Nurse can easily identify different size catheter with different color connectors.




PVC重头胃管 PVC Ryle's Stomach Tube

- ◆ 导管底端带不锈钢珠设计，便于导管在指定位置进行有效的留置。
Stainless steel balls are sealed in the tip end; it is easier to indwell the tube at a certain position.

产品编号 Product No.		规格 (Fr) Size(Fr)	色标 Color Code	长度 (mm) Length(mm)		刻度位置 (cm) Marked Position(cm)
Standard	Ryle's			Standard	Ryle's	
ST10P	ST10PR	10	黑色/Black	1200	1250	45/55 65/75
ST12P	ST12PR	12	白色/White			
ST14P	ST14PR	14	绿色/Green			
ST16P	ST16PR	16	橙色/Orange			
ST18P	ST18PR	18	红色/Red			
ST20P	ST20PR	20	黄色/Yellow			

PVC鼻饲管 PVC Nasogastric Tube


- ◆ 采用医用级PVC生产，管体透明度高。
Made of medical grade PVC, the tube is transparent and clear.
- ◆ X光造影线可供选择。
X-ray opaque line is available for choice.
- ◆ 不同规格采用不同色标区分。
Different sizes tubes are identified with color-coded connector.

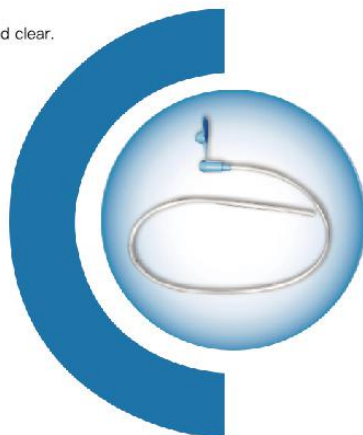

产品编号 Product No.		规格 (Fr) Size(Fr)	色标 Color Code	长度 (mm) Length(mm)	刻度位置 (cm) Marked Position(cm)
Without X-ray Line	With X-ray Line				
FT04PS	FT04PSX	4	紫色/Purple	400	14/15/16/20
FT05PS	FT05PSX				
FT05PL	FT05PLX	5	灰色/Gray	1000	45/55/65/75
FT06PS	FT06PSX				
FT06PL	FT06PLX	6	淡绿色 Light Green	400	14/15/16/20
FT08PS	FT08PSX				
FT08PL	FT08PLX	8	淡蓝色 Light Blue	400	14/15/16/20
FT10PL	FT10PLX				
		10	黑色/Black	1000	45/55/65/75

硅胶扁平开槽引流管 Silicone Flat Fluted Drain

- ◆ 100%采用医用级硅胶生产。
Made of 100% imported medical grade silicone.
- ◆ 扁平引流管由一段透明的硅胶圆管和一段白色扁管组成。内壁有槽状结构，防止因挤压或纠结引起的管体阻塞。
This Drain combines a transparent silicone round tube and a white flat drain. The inner wall of the implanted flat segment features multiple narrow ducts to prevent possible occlusion when the tube is pressed or kinked.
- ◆ 全开槽适应最佳引流选择。
Full fluted design is available for optimal drainage with X-ray opaque effectiveness throughout its entire implanted segment.
- ◆ 可使用不锈钢引导针进行手术穿刺放置，并用负压球收集引流液体，成套使用。

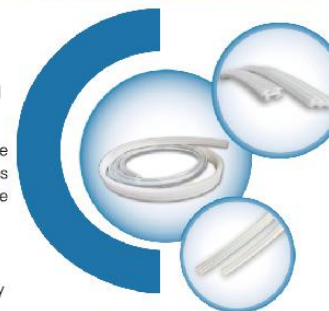


Using Trocar for insertion and combining with silicone Reservoir for fluids suction and collection.

产品编号 Product No.	扁平管宽度 Width of flat part (mm)	扁平管长度 Length of flat part (mm)
PFD083008	8	300
PFD112008	11	300
Remark	Length could be customized.	

硅胶圆形开槽引流管 Silicone Round Fluted Drain

- ◆ 100%采用医用级硅胶生产。
Made of 100% imported medical grade silicone.
- ◆ 带有X光造影线，用于闭合创伤引流手术。
With X ray opaque line, the round drain is used for closed wound drainage surgical.
- ◆ 管体内部有四个腔道，有利于更快的引流分泌物。
Four channel design is better for a quicker drainage of blood or secretion.
- ◆ 管体采用一体挤出工艺，无多余接口。术后移除引流管时，可减少患者痛苦。
The tube is extruded in one shot. With hubless design, it reduces patient's pain while removing the drain after surgical.

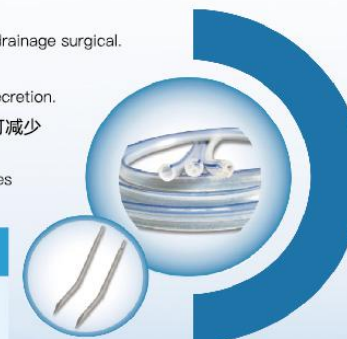


产品编号 Product No.	规格 Size(Fr)	外径 O.D.(mm)	长度 Length(mm)	备注 Remark
RFD1030	10Fr	3.3	1200	Length could be customized
RFD1530	15Fr	5.0	1200	
RFD1930	19Fr	6.3	1200	
RFD2430	24Fr	8.0	1200	

硅胶圆形打孔引流管 Silicone Round Perforated Drain


- 100%采用医用级硅胶生产。
Made of 100% imported medical grade silicone.
- X光造影线贯穿管身。
X-ray opaque line through the whole tube.
- 与负压引流球及引导针配合使用。
It is used together with reservoir and trocar.
- 管体透明，便于观察引流情况。
Transparent tube design makes drainage situation visible.

产品编号 Product No.	规格 (Fr) Size(Fr)	外径 O.D.(mm)
RPD10	10	3.3
RPD12	12	4.0
RPD14	14	4.7
RPD16	16	5.3
RPD18	18	6.0
RPD20	20	6.7
Remark	Length could be customized.	

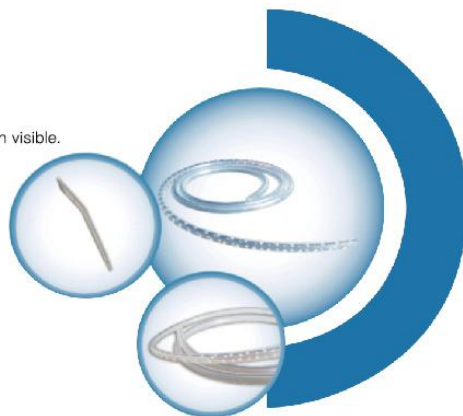

PVC圆形打孔引流管 PVC Round Perforated Tube


- 采用医用级PVC生产，管体透明度高。
Made of medical grade PVC, the tube is transparent and clear.
- X造影线贯穿管身。
X-ray opaque line through the whole tube.
- 端部32个侧孔设计，引流效率更高。
With 32 lateral eyes design on distal end, the drainage efficiency is much higher.
- 型号：10Fr-18Fr。
Size: 10Fr-18Fr.

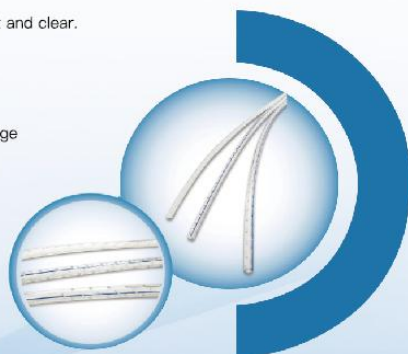

硅胶扁平打孔引流管 Silicone Flat Perforated Drain


- 100%采用医用级硅胶生产。
Made of 100% imported medical grade silicone.
- 扁平引流管由一段透明的硅胶圆管和一段白色扁平管组成。内壁有齿状结构，防止因挤压或纠结引起的管体阻塞。
This Drain combines a transparent silicone round tube and a white flat drain. The inner wall of the implanted flat segment features tooth-like to prevent possible occlusion when the tube is pressed or kinked.
- 扁平管3/4打孔，或全尺寸打孔，适应最佳引流选择。
3/4 or full perforation are available for optimal drainage with X-ray opaque effectiveness throughout its entire implanted segment.
- 可使用不锈钢引导针进行手术穿刺放置，并用负压球收集引流液体，成套使用。
Using Trocar for insertion and combining with silicone Reservoir for fluids suction and collection.

产品编号 Product No.	扁平管宽度 Width of flat part(mm)	扁平管长度 Length of flat part(mm)
FD0725	7	250
FD1030	10	300
FD1334	13	340
Remark	Length could be customized.	

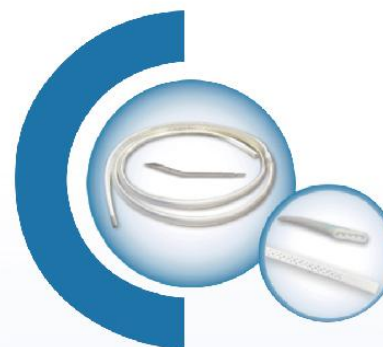

硅胶胸腔引流管 Silicone Chest Drain

- 100%采用医用级硅胶生产。
Made of 100% imported medical grade silicone.
- X光造影线贯穿管身。
X-ray opaque line through the whole tube.
- 型号：8Fr-36Fr。
Size: 8Fr-36Fr.

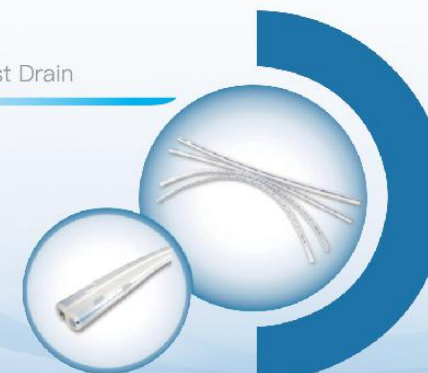

创伤引流类

WOUND DRAINAGE SERIES


创伤引流类

WOUND DRAINAGE SERIES

硅胶T型引流管(胆道引流管) Silicone T-shaped Drain (Kher T Tube)

- 100%采用医用级硅胶生产。
Made of 100% imported medical-grade silicone.
- 带X光显影线, 具有良好的生物相容性。
With X ray opaque line, and good bio-compatibility.

产品编号 Product No.	规格 (Fr) Size(Fr)	外径 O.D.(mm)
PTD101850	10	3.3
PTD121850	12	4.0
PTD141850	14	4.7

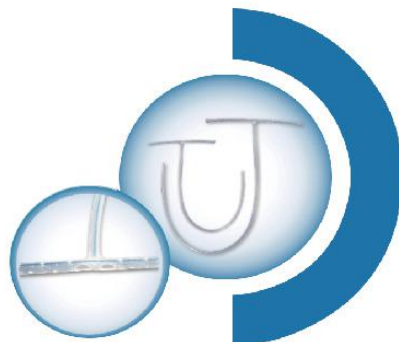

硅胶负压引流球 Silicone Reservoir / Bulb

- 采用进口医用级硅胶生产, 带止回阀。
Made of imported medical grade silicone, with anti-reflux valve.
- 负压球可以和各种类型的引流管配套使用。
Silicone Reservoir is used with different kind of drains.
- 加压球体使之产生负压, 将球体和引流管连接, 在密闭环境下, 球体内的负压可以稳定地抽吸血液或分泌物。
Squeeze the reservoir to generate negative pressure inside. Blood or secretion will be sucked inside the bulb due to negative pressure.



产品编号 Product No.	规格 (ml) Size(ml)	刻度位置 (ml) Marked Position(ml)	备注 Remark
SR100S	100	25/50/75/100	Clamp available
SR150S	150	25/50/75/100/125/150	
SR200S	200	50/100/150/200	
SR400S	400	100/200/300/400	
SR100ST	100	25/50/75/100	With extension tube
SR200ST	200	50/100/150/200	

硅胶引流套装 Closed Wound Drainage Kit

- 100%采用医用级硅胶生产。
Made of 100% imported medical-grade silicone.
- 该套装包含各型号负压球, 引流管和不锈钢引导针。
The kit contains different sizes reservoir, drainage tube and stainless steel trocar.
- 创伤引流套装用于引流伤口处的血液和多余液体, 从而减少血肿及消除引流死角。
Used to remove blood and excess fluid from wound, so as to prevent or minimize hematomas and eliminate dead space.
- 如有必要, 可与转换接头配用。
Connectors can be equipped with drains if required.




产品编号 Product	配件 Components	备注 Remark
Closed Wound Drainage Kit	100/150/200/400ml silicone reservoir flat/round fluted/perforated drain stainless steel trocar	Connectors available

不锈钢引导针

Stainless Steel Trocar


- 不锈钢材质, 尖端锋利, 针体光滑。
Made of stainless steel, with sharp pinpoint and smooth body.
- 适当的弯曲角度, 便于创伤穿刺。
With a proper angle, which is more convenience of puncture.
- 尾部光滑设计或阶梯设计可供选择。
With a smooth end or terraced end for choice.
- 可定做双端弯曲引导针。
Double bendable trocar could be customized.
- 尺寸: 10Fr / 15Fr / 19Fr, 尺寸可定做。
Size: 10Fr / 15Fr / 19Fr, size could be customized.

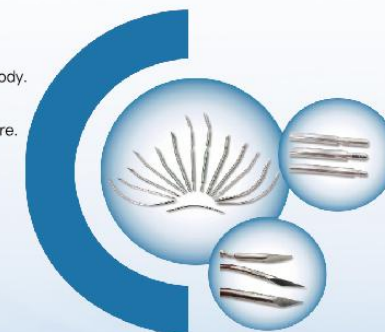

双腔硅胶导尿管

2-Way Silicone Foley Catheter

- 100%采用医用级硅胶生产，管体柔软透明。
Made of 100% medical grade silicone, the catheter is soft and clear.
- 球囊具有良好的均衡性和拉伸性，使用过程中更加安全、可靠。
Symmetrical balloon expands equally in all directions so as to perform its function safely and efficiently.
- 配备不同颜色的色标环以区分型号。
长度：400mm ± 10mm。
Equipped with different color rings to identify different sizes.
Length: 400mm ± 10mm.

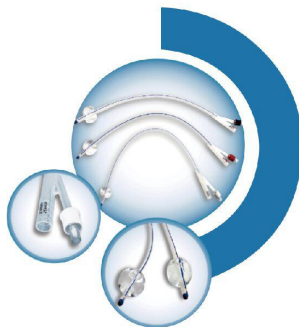


产品编号 Product No.	规格 (Fr) Size(Fr)	色标号 Color Code	球囊容量 (ml) Balloon Volume(ml)	备注 Remark
FC1210S2	12	橙色/白色/White	5-10ml	不带导丝 Without stylet
FC1410S2	14	绿色/Green		
FC1610S2	16	橙色/Orange		
FC1810S2	18	红色/Red		
FC2010S2	20	黄色/Yellow		
FC2210S2	22	紫色/Purple		
FC2410S2	24	蓝色/Blue		
FC1215S2	12	白色/White	5-15ml	
FC1415S2	14	绿色/Green		
FC1615S2	16	橙色/Orange		
FC1815S2	18	红色/Red		
FC2015S2	20	黄色/Yellow		
FC2215S2	22	紫色/Purple		
FC2415S2	24	蓝色/Blue		
FC1630S2	16	橙色/Orange	30ml	
FC1830S2	18	红色/Red		
FC2030S2	20	黄色/Yellow		
FC2230S2	22	紫色/Purple		
FC2430S2	24	蓝色/Blue		

双腔硅胶导尿管(儿童型)

2-Way Silicone Foley Catheter (Pediatric)

- 100%采用医用级硅胶生产，管体柔软透明。
Made of 100% medical grade silicone, the catheter is soft and clear.
- 球囊具有良好的均衡性和拉伸性，使用过程中更加安全、可靠。
Symmetrical balloon expands equally in all directions so as to perform its function safely and efficiently.
- 配备不同颜色的色标环以区分型号。长度：310mm ± 10mm。
Equipped with different color rings to identify different sizes. Length: 300mm±10mm.

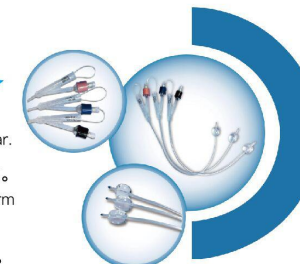


产品编号 Product No.	规格 (Fr) Size(Fr)	色标号 Color Code	球囊容量 (ml) Balloon Volume(ml)	备注 Remark
FC0603S2	6	粉色/Pink	3ml	带导丝 With stylet
FC0803S2	8	浅蓝色/Light Blue	3ml	
FC1003S2	10	黑色/Black		
FC0805S2	8	浅蓝色/Light Blue	3-5ml	
FC1005S2	10	黑色/Black		

双腔硅胶导尿管(弯头型/开口型)

2-Way Silicone Foley Catheter (Tiemann Tip/Open Tip)


- 100%采用医用级硅胶生产，管体柔软透明。
Made of 100% medical grade silicone, the catheter is soft and clear.
- 球囊具有良好的均衡性和拉伸性，使用过程中更加安全、可靠。
Symmetrical balloon expands equally in all directions so as to perform its function safely and efficiently.
- 导头可提供弯头型和开口型设计。长度：400mm ± 10mm
Both Tiemann tip and open tip design are available. Length: 400mm ± 10mm.

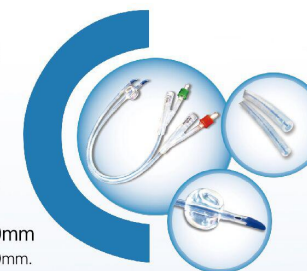

产品编号 Product No.	规格 (Fr) Size(Fr)	色标号 Color Code	球囊容量 (ml) Balloon Volume(ml)	备注 Remark
FC1215S2T	12	白色/White	5-10ml / 5-15ml	不带导丝 Without stylet
FC1415S2T	14	绿色/Green		
FC1615S2T	16	橙色/Orange		
FC1815S2T	18	红色/Red		
FC2015S2T	20	黄色/Yellow		
FC2215S2T	22	紫色/Purple		
FC2415S2T	24	蓝色/Blue		
FC2615S2T	26	粉红/Pink		

三腔硅胶导尿管 3-Way Silicone Foley Catheter

- ◆ 100%采用医用级硅胶生产，管体柔软透明。
Made of 100% medical grade silicone, the catheter is soft and clear.
- ◆ 球囊具有良好的均衡性和拉伸性，使用过程中更加安全、可靠。
Symmetrical balloon expands equally in all directions so as to perform its function safely and efficiently.
- ◆ 配备不同颜色的色标环以区分型号。
长度：400mm ± 10mm。
Equipped with different color rings to identify different sizes.
Length: 400mm ± 10mm.

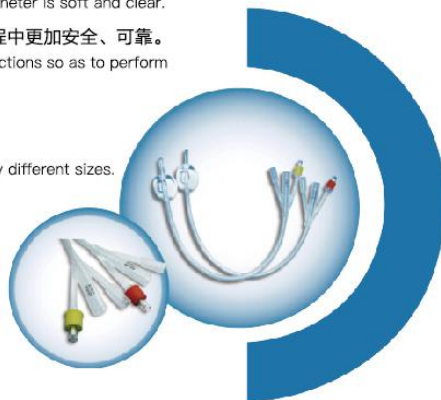


产品编号 Product No.	规格 (Fr) Size(Fr)	色标号 Color Code	球囊容量 (ml) Balloon Volume(ml)	备注 Remark
FC1615S3	16	橙色/Orange	5-15ml	不带导丝 Without stylet
FC1815S3	18	红色/Red		
FC2015S3	20	黄色/Yellow		
FC2215S3	22	紫色/Purple		
FC2415S3	24	蓝色/Blue		
FC1630S3	16	橙色/Orange	30ml	
FC1830S3	18	红色/Red		
FC2030S3	20	黄色/Yellow		
FC2230S3	22	紫色/Purple		
FC2430S3	24	蓝色/Blue		

单腔乳胶导尿管

Latex Urethral Catheter

- ◆ 采用天然乳胶生产，表面做硅胶涂层处理。
Made of natural latex, with silicone coated.
- ◆ 管体有红色和黄色可供选择。
Both red color and yellow color are available.
- ◆ 长度：400mm ± 10mm。
Length: 400mm ± 10mm.
- ◆ 尺寸：6Fr-30Fr。
Size: 6Fr-30Fr.

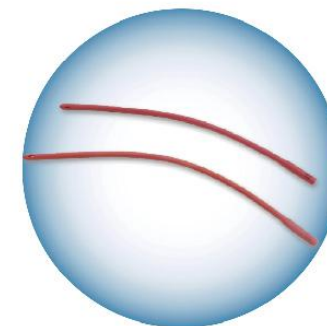

双腔乳胶导尿管 (弯头型)

2-Way Latex Foley Catheter (Tiemann Tip)

- ◆ 采用天然乳胶生产，表面做硅胶涂层处理。
Made of natural latex, with silicone coated.
- ◆ 弯头设计，更容易置入。
It is easier to be inserted with a Tiemann tip design.
- ◆ 长度：400mm ± 10mm。
Length: 400mm ± 10mm.




产品编号 Product No.	规格 (Fr) Size(Fr)	色标号 Color Code	球囊容量 (ml) Balloon Volume(ml)	备注 Remark
FC1210L2T	12	白色/White	5-10ml	不带导丝 Without stylet
FC1410L2T	14	绿色/Green		
FC1615L2T	16	橙色/Orange	5-10ml 5-15ml 30ml	
FC1815L2T	18	红色/Red		
FC2015L2T	20	黄色/Yellow		
FC2215L2T	22	紫色/Purple		
FC2415L2T	24	蓝色/Blue		
FC2615L2T	26	粉红/Pink		

双腔乳胶导尿管

2-Way Latex Foley Catheter

- 采用天然乳胶生产，表面做硅胶涂层处理。
Made of natural latex, with silicone coated.
- 有硬阀和软阀可供选择。
Both plastic valve and rubber valve are available.
- 长度：儿童270mm ± 10mm，
成人400mm ± 10mm。
Length: Pediatric 270mm ± 10mm,
Adult 400mm ± 10mm.

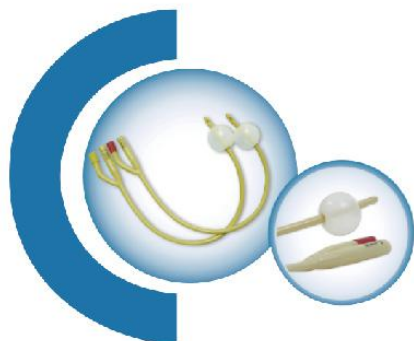


产品编号 Product No.	规格 (Fr) Size(Fr)	色标号 Color Code	球囊容量 (ml) Balloon Volume(ml)	备注 Remark
FC0803L2	8	黑色/Black	3-5ml	带导丝 With stylet
FC1003L2	10	灰色/Grey		
FC1215L2	12	白色/White	5-15ml	不带导丝 Without stylet
FC1415L2	14	绿色/Green		
FC1615L2	16	橙色/Orange		
FC1815L2	18	红色/Red		
FC2015L2	20	黄色/Yellow		
FC2215L2	22	紫色/Purple		
FC2415L2	24	蓝色/Blue		
FC2615L2	26	粉红/Pink		
FC1630L2	16	橙色/Orange		
FC1830L2	18	红色/Red		
FC2030L2	20	黄色/Yellow		
FC2230L2	22	紫色/Purple		
FC2430L2	24	蓝色/Blue		
FC2630L2	26	粉红/Pink		

三腔乳胶导尿管

3-Way Latex Foley Catheter


- 采用天然乳胶生产，表面做硅胶涂层处理。
Made of natural latex, with silicone coated.
- 有硬阀和软阀可供选择。
Both plastic valve and rubber valve are available.
- 长度：400mm ± 10mm
Length: 400mm ± 10mm.

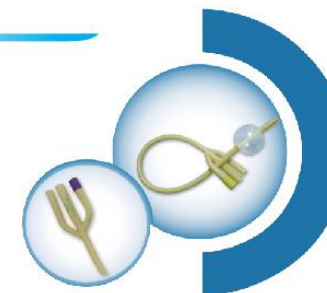

产品编号 Product No.	规格 (Fr) Size(Fr)	色标号 Color Code	球囊容量 (ml) Balloon Volume(ml)	备注 Remark
FC1630L3	16	橙色/Orange	30ml	不带导丝 Without stylet
FC1830L3	18	红色/Red		
FC2030L3	20	黄色/Yellow		
FC2230L3	22	紫色/Purple		
FC2430L3	24	蓝色/Blue		
FC2630L3	26	粉色/Pink		

直肠管 Rectal Tube

- 采用医用级PVC生产，管体透明度高。
Made of medical grade PVC, the tube is transparent and clear.
- 长度：400mm ± 10mm。
Length: 400mm ± 10mm.
- 尺寸：24Fr-36Fr。
Size: 24Fr-36Fr.

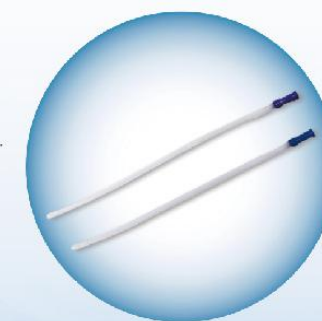


PVC导尿管 PVC Nelaton Catheter

- 采用医用级PVC生产。
Made of medical grade PVC.
- 圆润的侧孔和头端处理，减少对患者黏膜的损害。
Smooth side eyes and closed distal tip design reduce patient's pain during surgical.

亲水性涂层型PVC导尿管 Hydrophilic Coated Nelaton Catheter


- 采用医用级PVC生产。
Made of medical grade PVC.
- 圆润的侧孔和头端处理，减少对患者黏膜的损害。
Smooth side eyes and closed distal tip design reduce patient's pain during surgical.
- 可提供弯头型设计。
Tiemann tip design is available.
- 导管表面最亲水涂层处理，遇水后极大增加润滑度，减少患者痛苦的同事更方便置入。
The surface of catheter is hydrophilic coated. It becomes extremely smooth while using with normal saline. It reduce the patient's pain and is much easier to be inserted.
- 提供带水包和不带水包型供客户选择。
Both kits with and without water sachet are available.

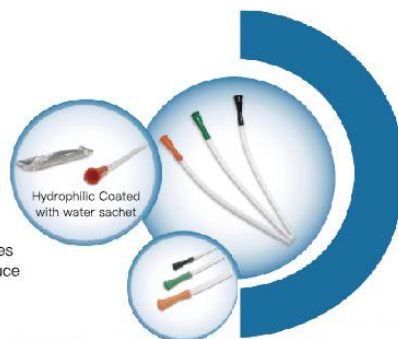

产品编号 Product No.	规格 (Fr) Size(Fr)	色标号 Color Code	长度 (mm ± 10mm) Length (mm ± 10mm)	备注 Remark
NC08PF	8	浅蓝色/Light blue	200	女用/Female
NC10PF	10	黑色/Black	200	女用/Female
NC12PF	12	白色/White	200	女用/Female
NC14PF	14	绿色/Green	200	女用/Female
NC16PF	16	橙色/Orange	200	女用/Female
NC08PM	8	浅蓝色/Light blue	400	男用/Male
NC10PM	10	黑色/Black	400	男用/Male
NC12PM	12	白色/White	400	男用/Male
NC14PM	14	绿色/Green	400	男用/Male
NC16PM	16	橙色/Orange	400	男用/Male
NC18PM	18	红色/Red	400	男用/Male
NC20PM	20	黄色/Yellow	400	男用/Male

一次性测温导尿管 Disposable Foley Catheter with Temperature Probe

- 产品供一次性使用。
This catheter is designed for single use only.
- 产品用于在给患者导尿的同时，测量患者膀胱内尿液的温度。
It is used for detecting urine temperature inside patient's bladder while urine drainage.
- 温度探头置于导尿管尖端和球囊之间。
The detector is indwelled between tip and balloon of catheter.

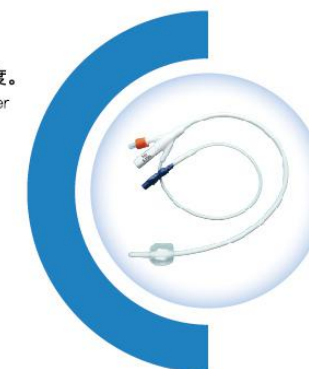

产品编号 Product No.	产品型号 (Fr) Size(Fr)	球囊容积 (ml) Balloon Volume(ml)	产品长度 (mm) Length(mm)
FC0803ST	8	3	358
FC1010ST	10	5-10	358
FC1210ST	12	5-10	458
FC1410ST	14	5-10	458
FC1630ST	16	15-30	458
FC1830ST	18	15-30	458
FC2030ST	20	15-30	458
FC2230ST	22	15-30	458
FC2430ST	24	15-30	458

宫颈扩张球囊 Cervical Ripening Balloon


在宫颈条件不佳时，富善宫颈扩张球囊为患者提供了一种简单促使宫颈成熟的方法，同时没有其他促宫颈成熟方法的潜在副作用。The FUSHAN Cervical Ripening Balloon offers a simple method for cervical ripening in unfavourable cervical conditions, without the potential side effects of other ripening methods.

对宫颈条件不佳的患者实施引产是一个颇具挑战性的产科治疗措施，而促宫颈成熟措施的实施可能会使结局更加复杂。某些药物治疗（如前列腺素）不仅可能会产生副作用和导致引产失败率上升，而且最终仍可能需借助剖腹产结束妊娠。

It's challenging to induce labour in patients with unfavourable cervical conditions. Such inductions may be complicated by the cervical ripening method that you use. Certain medical therapies, such as the use of prostaglandins, may have side effects and high failed induction rates while still resulting in undesired caesarean section deliveries.

富善宫颈扩张球囊能自然地，渐进式扩张宫颈并易化引产。宫颈的成熟与扩张通过球囊在宫颈内口和外口提供温和、持久的扩张力而实现。

The FUSHAN Cervical Ripening Balloon is engineered to naturally and gradually dilate the cervix and facilitate labour induction. Ripening and dilation are accomplished by the balloons' gentle and constant pressure at the level of the cervix from both the internal and external ostia.

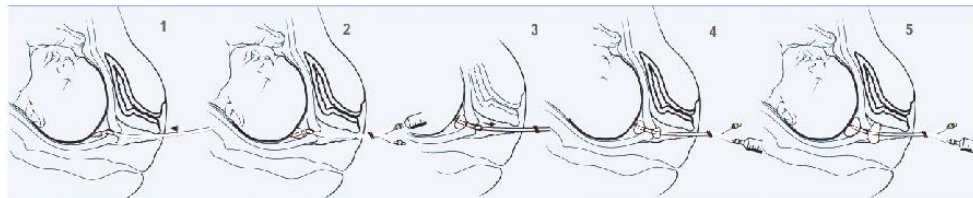

1、通过宫颈放入宫颈扩张球囊，使两个球囊均进入宫颈管。

The Cervical Ripening Balloon through the cervix until both balloons have entered the cervical canal.

2-3、用40ml生理盐水充盈子宫球囊。充盈后，将球囊往后拉至子宫球囊紧贴宫颈内口。

Inflate the uterine balloon with 40 ml of saline. Once the uterine balloon is inflated, pull the device back until the balloon abuts the internal cervical os.

4、此时可见阴道球囊位于宫颈外口，用20ml生理盐水充盈阴道球囊。

You should see the vaginal balloon outside the external cervical os. Inflate the vaginal balloon with 20 ml of saline.

5、将球囊置于宫颈两侧后，充入生理盐水—每个球囊最多80ml，球囊放置后应计时，在进入分娩活跃期前球囊留置不得超过12小时。欲了解采用富善宫颈扩张球囊进行产程管理的更多信息，请参阅产品使用说明书。

Once the balloons are situated on either side of the cervix, add saline—a maximum of 80 ml per balloon. Time the placement of the balloon so that it is in place no longer than 12 hours before active labour is induced. For more information on labour management using the FUSHAN Cervical Ripening Balloon, please refer to the Instructions for Use.

