


1


**CLEAN
&
DRY**

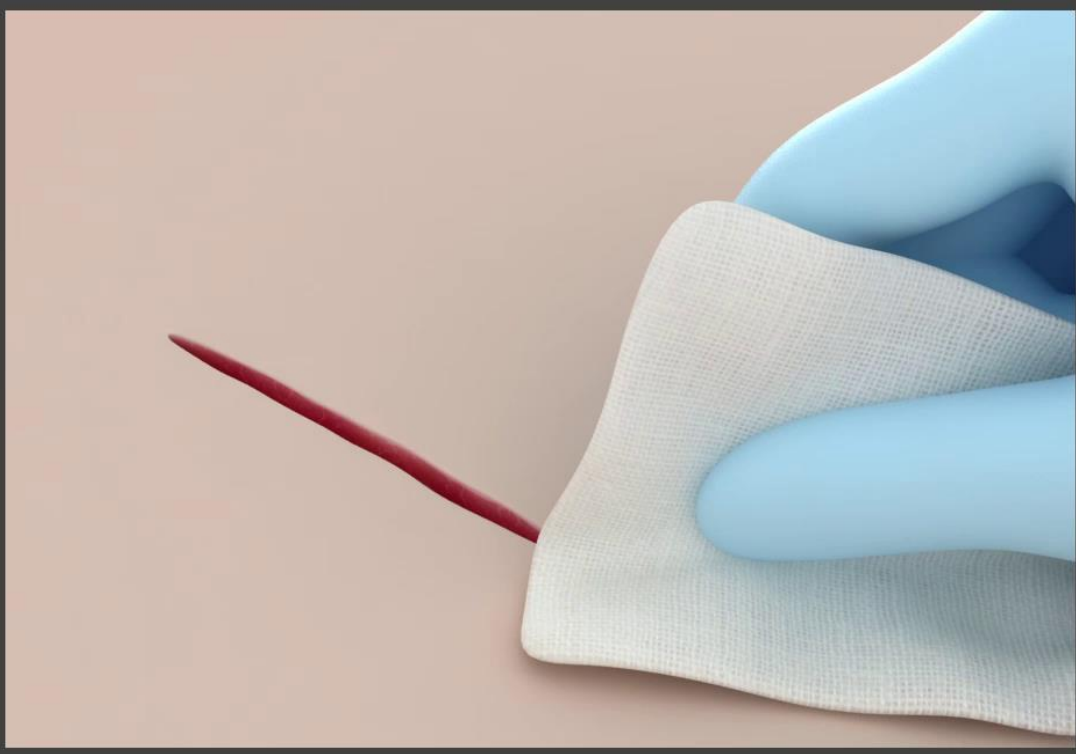

After the tube has been placed, clean and dry the area around it for product to apply.

2


**REMOVE
RELEASE
PAPER**

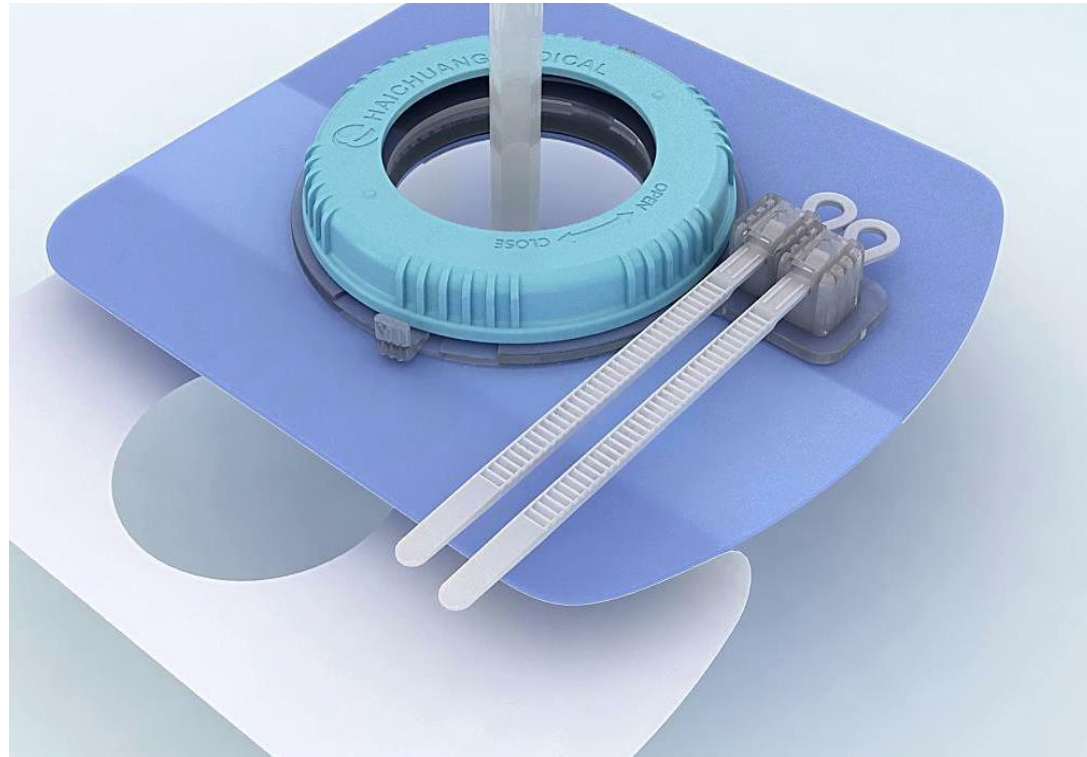

Through the tube and remove #1 release paper, do not remove #2 release paper at this step

3


**CENTER
LOOPIX**

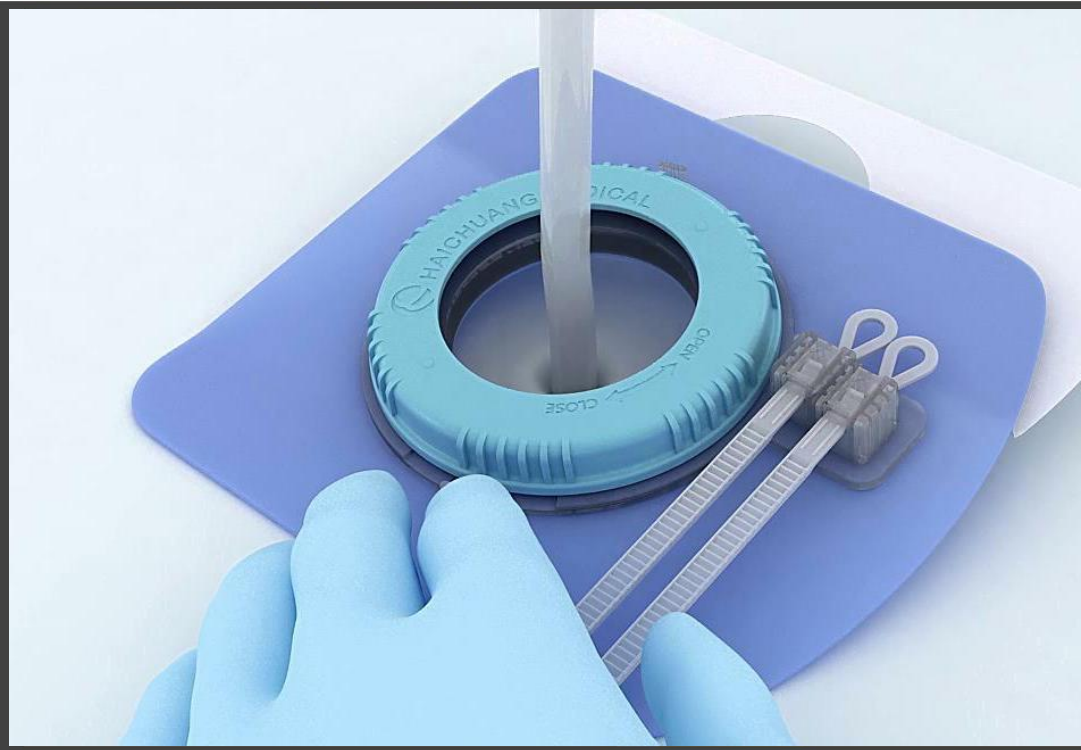

Center the loopix and attach the adhesive pad to the skin, then release the #2 film, make sure the pad is fully attached to the skin.

4

**PLACE
THE
GAUZE**

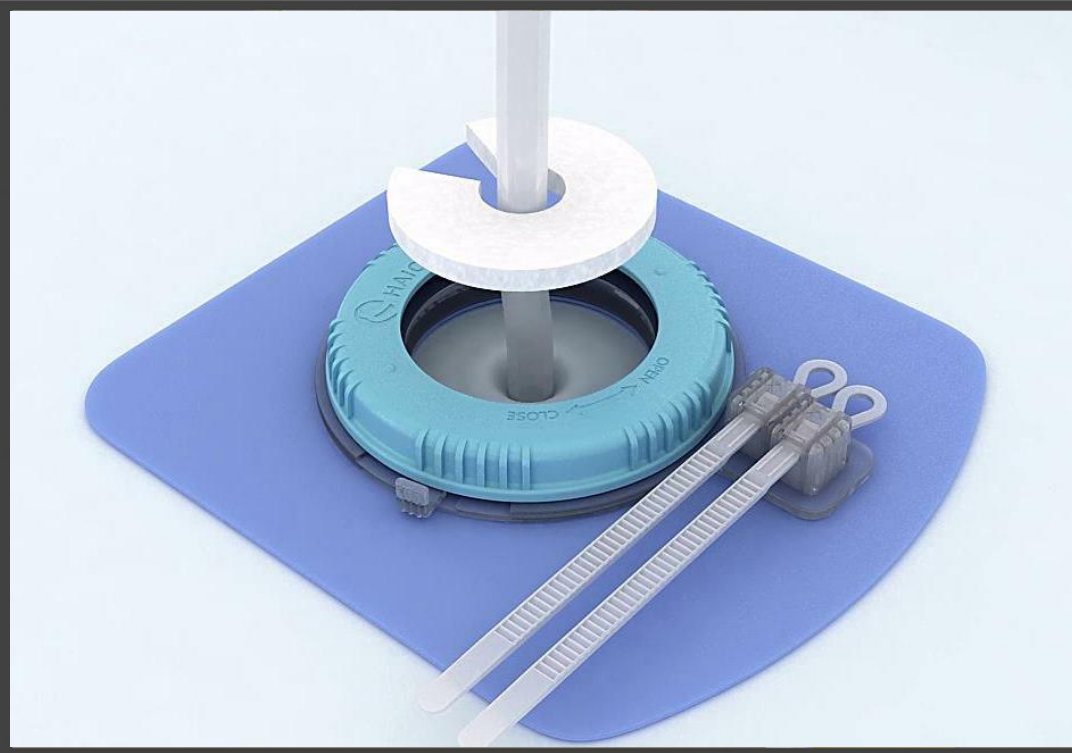

Place the C-shape gauze around the tube to absorb the exudate.

5


ROTATE
THE
"LOOP"



Rotate the loop (blue ring) clockwise, the hidden hoops will tightening tube as rotating.

6


**DOUBLE
SECURE**

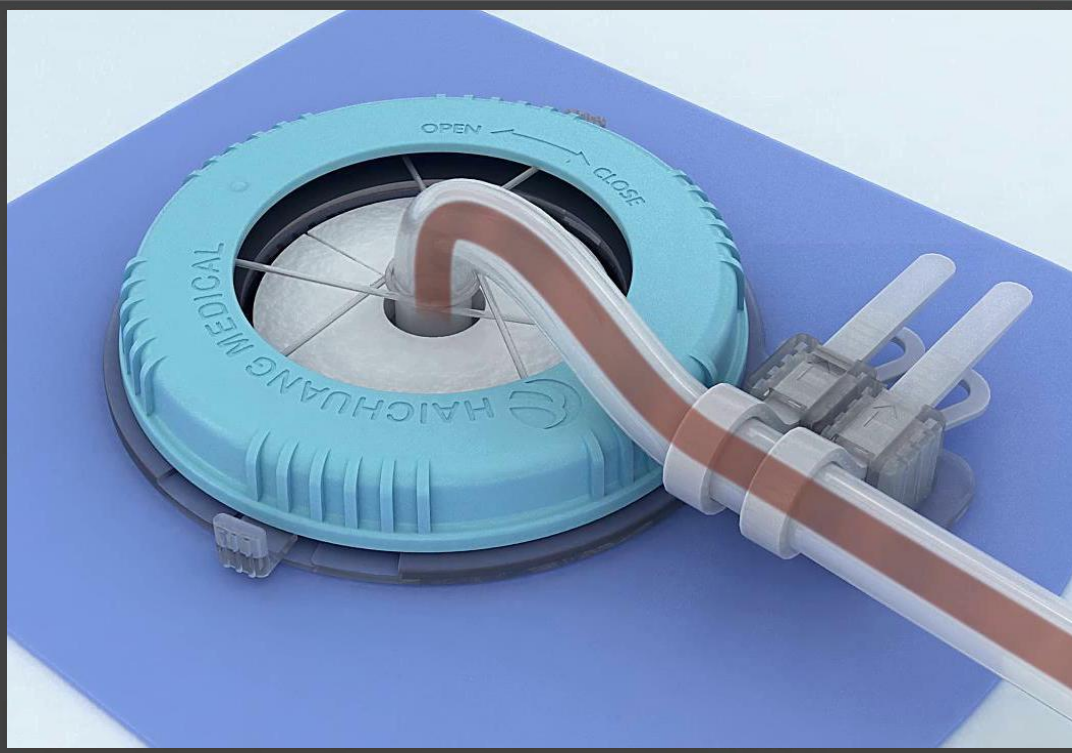

Fasten two cable ties to stabilize the direction of tube.

7

ADJUST
THE
"LOOP"

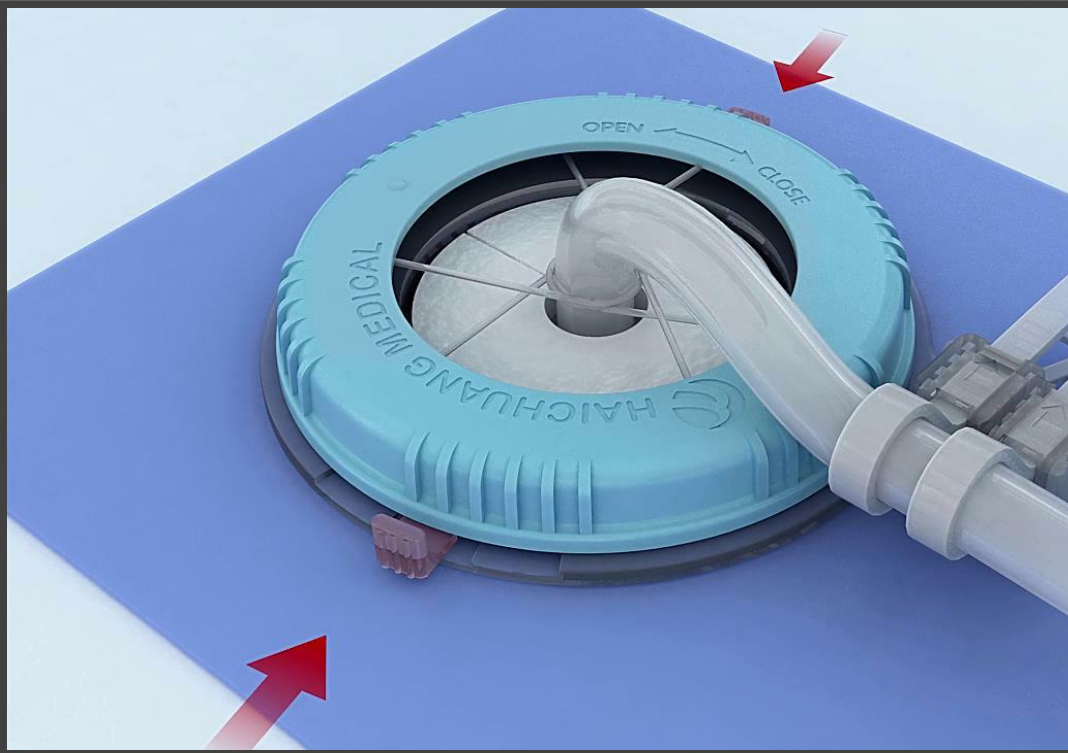

Hold the buttons pointed by red arrows at same time, and rotate count-clockwise to release the hoops to adjust/change/remove the tubes.