Gel Positioner Factory

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Operation position pads series

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Functional Pad Series

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Memory Cotton Products Series

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Undertake development of customized projects

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Contraindications:

- 1、It is forbidden to be used for damaged parts of the body surface that require air permeability during treatment.
- 2、Disabled for patients with contact allergy to polyurethane materials.
- 3、Those who are overweight and require prone position surgery are prohibited.

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Precautions:

- 1、Avoid contact with hard and sharp objects;
- 2、Do not use corrosive and chlorine-containing disinfectant cleaners to clean the surface of the mat;
- 3、It should be placed flat and dustproof in normal storage; avoid ultraviolet radiation;
- 4、Avoid pushing the surgical position pad under the patient's use site with force to ensure the smoothness of the contact surface between the pad and the body;
- 5、It is recommended that if the product is used for a long time (especially in the prone position), the surface of the product should be padded with an accessory sheet or a surgical towel to ensure the ventilation of the contact part, and the vulnerable parts should be massaged for 5 minutes every 3 hours.

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Cleaning and Disinfection Methods:

- 1、For the dirt on the surface of the mat, it can be wiped clean with water or neutral detergent, and disinfected with 75% alcohol;
- 2、Wiping with iodine-containing disinfectant may stain the product, but it will not affect the service life of the product;
- 3、Can not be sterilized and fumigated at high temperature and high pressure;
- 4、Do not soak in disinfectant for a long time;
- 5、Avoid disinfection with ultraviolet radiation.

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Product process comparison

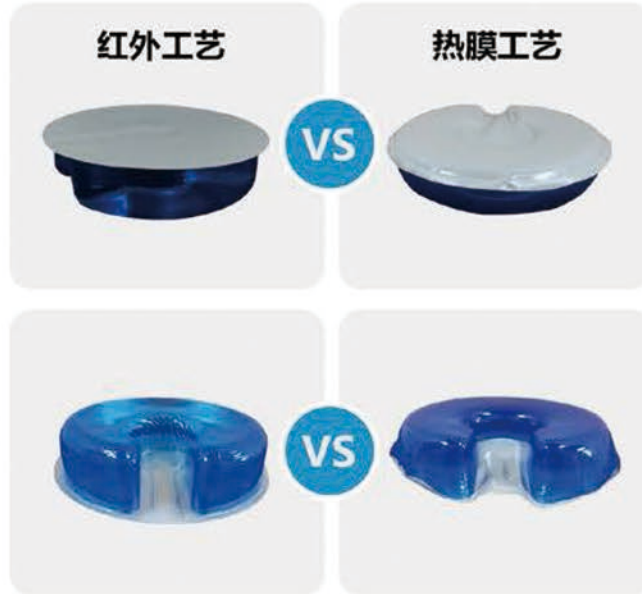
Infrared high-energy long wave shaping technology

Hot Film technology

Adopting **infrared high-energy long wave shaping technology**.

The product is well formed and meets the design requirements.

The bottom is flat and can fit tightly with the surgical bed surface, providing better stability. Never deform, never harden, and if there is any damage, it is just a small wound and will not burst.

The product adopts a hot film shaping process. (Customizable, low price)

Insufficient shaping, the product will rebound after demolding, and the bottom edge will bulge, which **cannot fully adhere to the bed surface and has poor stability**.

It may deform and harden in the later stage. **The damaged area will continue to burst.**

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Characteristics of gel surgery body cushions:

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It is molded in one time with advanced and unique gel material. It is ergonomically designed and has good elasticity to provide long-lasting support.

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It is suitable for all skin types, the weight is evenly distributed on the gel, and the fluid gel effectively disperses the pressure and provides excellent pressure release.

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The position is comfortable, the fixation is firm and there is no movement, and the surgical field is exposed clearly. The operation is convenient.

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It does not store heat and prevents the production of acne.

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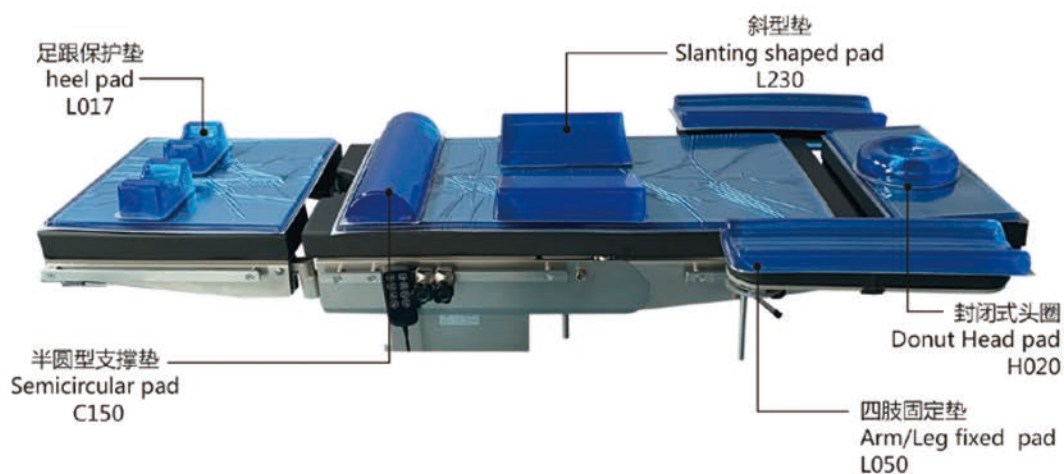
It does not contain latex, has no allergic reaction, is non-conductive, non-flammable, does not absorb odor, and can pass X-ray.

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Easy to disinfect, easy to clean, with good flexibility and pressure resistance. Very satisfied and suitable for long-term, high-intensity surgery needs.

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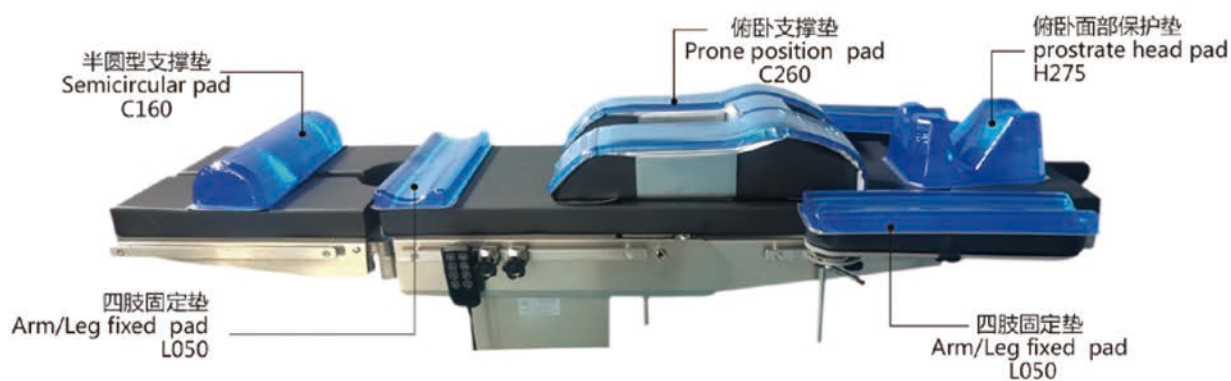
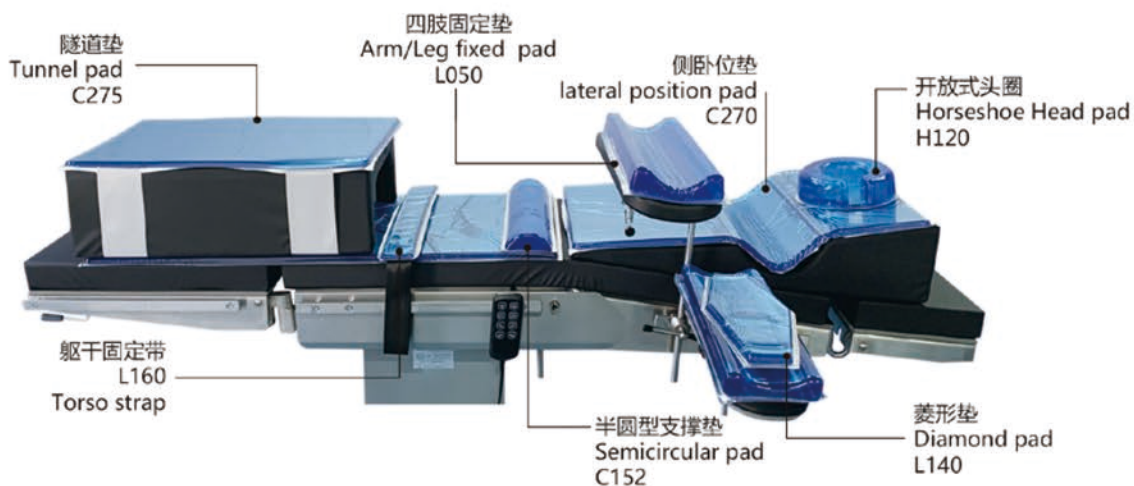
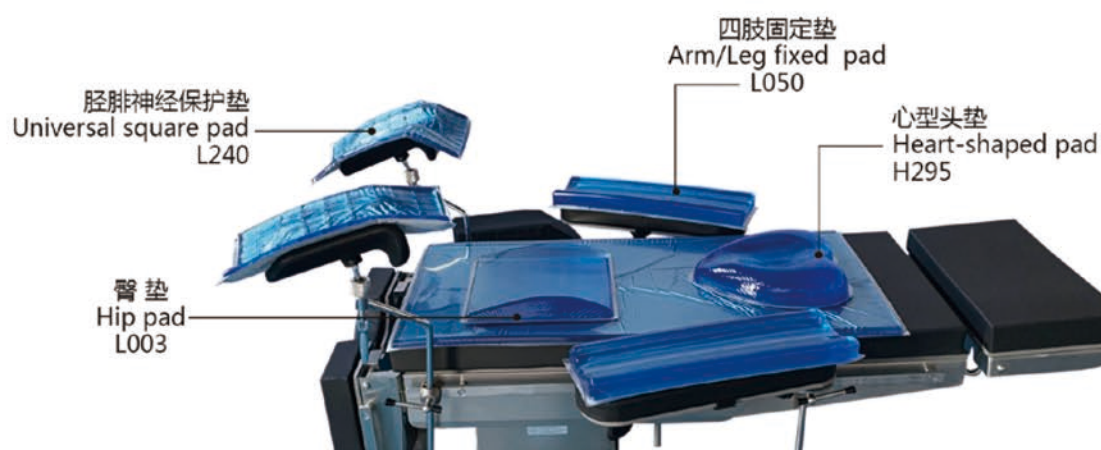
Features : It is used for the support, fixation and protection of the head and neck during the supine position operation, which can reduce the pressure of the compression point.

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Features : It is used for the support and protection of the head and face during lateral and prone operations, and is convenient for anesthesia intubation.

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Features : It is used to support, fix and protect the head and neck during supine surgery.

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Features：It is used for the support, fixation and protection of the head and neck during the supine position operation, which can reduce the pressure of the compression point.

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Features：It is specially designed for the support and protection of the head and face in the prone position to facilitate anesthesia intubation, and the height can be adjusted according to the actual situation. The bottom mirror can observe the patient's condition in real time during anesthesia.

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Features： Designed for support of the face during prone position surgery, convenient for anesthesia intubation, keeping breathing smooth.

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Features： ISupport and fixed protection for head and neck during supine surgery.

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Features: For the support and fixation of the head and neck during ophthalmic surgery.

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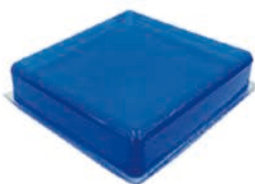
Features: It has strong versatility and is widely used. It can reduce the pressure of pressure points and prevent the formation of pressure ulcers.

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Features： Support and fixed protection for head and neck during supine surgery

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Features： Fixed on the shoulder rest of the operating table, Protection for shoulder compression points.

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Features： For thyroid and neck surgery, strengthen the support and protection of the back of the neck. Can fully expose the.

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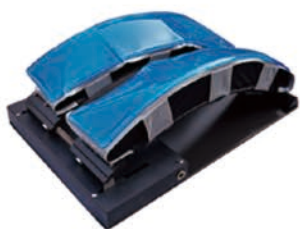
功能：专为俯卧位手术时减轻胸部压力，维持通畅的呼吸循环功能。
Features： It is designed to relieve chest pressure during prone position and maintain a smooth breathing cycle.

型号/model	规格/specifications
C264	66*33-40*22cm
C267 (不锈钢)	70*25-45*21-28cm

功能：专为俯卧位手术时，特别是脊柱手术时使用，可根据术中情况调节宽度及拱度，更好的暴露手术视野。
Features： It is specially designed for prone surgery, especially for spinal surgery. The width and camber can be adjusted according to the situation during the operation, so as to better expose the surgical field of vision.

型号/model	规格/specifications
C266	75*50*25-33cm

功能：支架为碳纤维材质，可透X光，多用于透视下的俯卧位手术，可根据术中情况调节宽度及拱度。
Features： The stent is made of carbon fiber and can pass X-rays. It is mostly used for prone surgery under perspective.

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Features： A special protective pad for prone surgery reduces the pressure on the chest and abdomen and provides protection for the compression points of the nerve tissue.

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Features：It is used in the prone position to support the chest and hips, maintain a good breathing cycle, and avoid pressure sores. It can also be used in the supine position to support the shoulders and chest.

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Features： It is used to protect the lower limbs during lateral decubitus surgery and prevent pressure ulcers. Suitable for people with heavier weight

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Features： It is designed for use in lateral position surgery to relieve pressure on the shoulders and arms and protect nerve tissue.

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Features：It is used to protect the compression point during surgery, relieve pressure on the shoulders, chest and abdomen, and protect the nerve tissue.

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Features: It is used to protect the compression point during surgery, relieve pressure on the shoulders, chest and abdomen, and protect the nerve tissue.

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Features：It has strong versatility and wide range of functions. It relieves pressure on compression points and prevents pressure ulcers from forming.

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Features：Used in baby incubators to reduce pressure and prevent the formation of pressure ulcers. Double-layer light transmission can be achieved during blue light treatment. Specifications and dimensions can be customized according to customer requirements.

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Features: In the lithotomy position, it is used for the popliteal fossa to protect the tibiofibular nerve and reduce the pressure. The deep groove design is more conducive to the.

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Features: For the immobilization and protection of the calf in the supine position

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Features: SProtect the fixed torso and joints for easy surgery and avoid injury.

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Features：It has strong versatility and is widely used. It can reduce the pressure of pressure points and prevent the formation of pressure ulcers.

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Features : it is used to protect the compression points of the limbs and knee joints, and the concave design can better fix it.

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Features : Decompression protection for upper extremities

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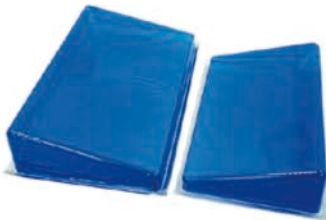
Features : The supine position is used to raise the lower limbs and keep the lower limbs physiologically curved. It is suitable for lower limb surgery, traction after tibia, fibula and femur fractures. Separate upper and lower support and protection for legs in lateral position

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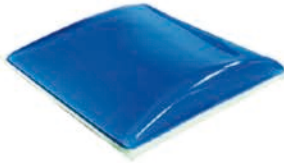
Features : It is suitable for supine, lateral and prone operations, and is mainly used to support and protect the waist, liver and spleen during supine operations.

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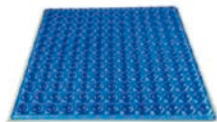
Features：It is used to protect the buttocks and sacrum during supine surgery, which can reduce the pressure on the compression point and prevent the formation of pressure ulcers.

型号/model	规格/specifications
L008	40*40*1.8cm
L009	40*40*2cm
L004	43*35*5cm

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Features：For the protection of the hip and sacrum when sitting or lying down after surgery.

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Features： When used in the prone position, the fixed protection of the knee joint prevents the formation of pressure ulcers.

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Features： Used to protect the heel and ankle.

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Features：Used in operating beds or hospital beds to reduce pressure on compression points, increase blood circulation, and avoid pressure ulcers.

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Features： During orthopedic traction, place it on the perineum to prevent bedsores.

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Features： Used in orthopaedic traction boots to wrap and protect the entire foot to prevent foot injury.

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Features： Hardware and gel are integrally formed. Instead of traditional hard products, it can be directly connected to the operating table to provide softer protection for compression points.

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