

WB-005 Condom length measuring ruler

WB-005 The condom length measuring ruler is suitable for measuring the length of condom, which meets the technical requirements and test methods of natural latex rubber condom (GB7544-2009).

Professional Technology

- Full angle multidirectional scale value display, convenient for customers to test in different directions.
- The scale value is engraved and colored, with long wear-resistant service life.



Testing Principle

Unfold the condom and stretch it gently twice, but the stretching length is not more than 20 mm, then put it on the round bar measuring ruler, let it drop freely by its own weight, and record the minimum length of the opening end on the scale, accurate to mm.

Testing Standard

The instrument conforms to many national and international standards: GB 7544-2009 Technical requirements and test methods of natural latex rubber condoms.

Application Areas

Basic Application	Suitable for measuring the length of various condoms.
	Suitable for measuring the length of various rubber gloves and balloon products.

Technical Parameter

Item	Model	WB-005
Measuring range		0-200mm
Graduation value		1.0mm
Outline size		Pedestal Φ 150mm * Height 230mm

Product configuration

Standard configuration	Mainframe
------------------------	-----------

FAQ

1. Are you a manufacturer or a trading company ?

A: We are a professional manufacturer of test instruments. And we have our own subsidiary company for trading. All the import and export business will be handled by our own trading company.

2. Where is your factory located ? How can I visit there ?

A: Our factory is located in Jinan City, Shandong Province, about 1.5 hours by train from Beijing.

3. How to support me if i don't know how to use it?

A: Free training within the warranty by tel and mail or skype. And we have professional operation manual and operation video. We also can let our technician to customer's company to provide training and support if customer request.

4. If machines wearing parts need changing, what should we do?

A: We can offer the wearing parts all the time, and offer video to show how to change it.