

# User Manual

【Item name】 NPWT disposable canister

【Model /specification】

Specification: 140ml, 500ml, 1000ml.

【Structure/composition】 NPWT disposable canister is a combination of a drainage bottle with a solidifier, double lumen tubing(90cm), and a Y connector. It is non-sterile and is meant to be used with a welsuc negative pressure wound therapy device. Combined with a sensor that alerts you when the tank is full. It has a hermetic seal and a teflon (waterproof moisture-permeable film) that can keep pressure under different conditions and prevent fluids from entering the vacuum pump unit.

【Scope of application】 NPWT disposable canister is indicated for to be used with welsuc negative pressure wound therapy device to collect the fluids such as excess exudates, infections material and tissue debris from the wound.

Performance:

1. The error between each marked volume scale on the canister body and the actual volume shall not exceed  $\pm 10\%$ .
2. NPWT disposable canister should be able to withstand negative pressure.

【Usage】

1. Unpack the NPWT disposable canister and check it in the complete situation.
2. Push the NPWT disposable canister into the vacuum pump until the disposable canister clicks into the place in the locking hole. And make sure they are well tightly connected.
3. Keep the vacuum pump in working condition, observe the entire negative pressure wound therapy system, and check for leak indicates in the touch screen and whether the entire negative pressure wound therapy system is working properly.
4. Pay attention to the amount of the fluids collected in the disposable canister, please replace in time.

【Contraindications】 Not clear yet.

【Precautions, warnings and prompt instructions】

1. NPWT canister is a disposable product, don't reuse it.
2. It is prohibited to use after the validity period;
3. Check the packaging carefully before use, if it is damaged or leaking, it must not be used.

【Explanation of the graphics, symbols, abbreviations etc. used in the label】



No Use Twice



Do not use if package is damaged

【Storage conditions】 Keep away from light, airtight, and store at room temperature.

【Validity period】 3 Years.

【LOT/MFG/EXP】 See the bottom of package box

【Manufacturer】 Guangzhou Rainhome Pharm & Tech Co., Ltd

【Factory address】 Room 201,2F,5F,Building 6,No.10 Yongsheng Road, Huangpu District,Guangzhou City, Guangdong Province, P.R.China.

Zip code: 511356

Tel: +86-20-26239185

Fax: +86-20-82086261

Website: [www.rainhomedical.com](http://www.rainhomedical.com)

【Last update】 17-06-2024

